



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number <i>80-006690</i>	Instrument Location <i>MNAF</i>
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>22 Jan 19</i>
---	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>[Signature]</i>	Certified Date <i>24 Jan 19</i>
---	------------------------------------


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
01/22/2019 08:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:27
02 Std. Gas	0.080	08:27
03 Room Air	0.000	08:27
04 Std. Gas	0.081	08:28
05 Room Air	0.000	08:28
06 Std. Gas	0.081	08:29
07 Room Air	0.000	08:29

Lot No = 19817080A4
Cyl No = 8
Exp Date = 09/05/2019
County = 64 Oper No. = 132026



Operator Signature
JEREMY HOMAN

Remarks: *Install*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
01/22/2019 08:30

Test	AC	Time
01 Room Air	RFI*	08:32
02 Room Air	0.000	08:33

*Invalid Test
Inhibited - RFI

Sub Name = HOMAN, JEREMY WAYNE
Sub DOB = 07/08/1992
Sub Sex = Male Weight = 160
Test = OTH Cit = N
Dr. Lic. = IL/H55043992194
Lot No = 19817080A4
Cyl No = 8
Expiration Date = 09/05/2019
County = 64 Oper No. = 132026



Operator Signature
JEREMY HOMAN

Remarks: Install

Form 106-I8000