



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) JEFFREY A. BAUSKE		
Location RELETTE COUNTY SD	Intoxilyzer® Serial Number 80-006511	
Gas Lot Number 13518080 A6	Gas Cylinder Number 33	Gas Expiration Date 08/05/2020

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.083 AC
 - B. 0.083 AC
 - C. 0.083 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jeffrey A. Bauske</i>	Date 03/05/19
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ehr</i>	Date 3/6/2019


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006511
Location = ROLL 8164.14.00 09/16
03/05/2019 08:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	08:04
02 Std. Gas	0.083	08:05
03 Room Air	0.000	08:05
04 Std. Gas	0.083	08:06
05 Room Air	0.000	08:06
06 Std. Gas	0.083	08:07
07 Room Air	0.000	08:07

Lot No = 13518080A6
Cyl No = 33
Exp Date = 08/05/2020
County = 40 Oper No. = 120501



Operator Signature
JEFFREY BAUSKE

Remarks:

Form 106-I8000