



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Wesley Straight</i>		
Location <i>Coop</i>	Intoxilyzer® Serial Number <i>80-006671</i>	
Gas Lot Number <i>13518080A6</i>	Gas Cylinder Number <i>25</i>	Gas Expiration Date <i>8/5/20</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

ACA TEST SUBMITTED. BOX NOT CHECKED. CEE

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

*CEE
3/22/19*

A. 0.082 AC

B. 0.082 AC

C. 0.082 AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>3/18/19</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>3/22/19</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = COOP 8164.14.00 09/16
03/18/2019 17:37

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:38
02 Std. Gas	0.082	17:38
03 Room Air	0.000	17:38
04 Std. Gas	0.082	17:39
05 Room Air	0.000	17:39
06 Std. Gas	0.082	17:40
07 Room Air	0.000	17:40

Lot No = 13518080A6
Cyl No = 025
Exp Date = 08/05/2020
County = 20 Oper No. = 061123

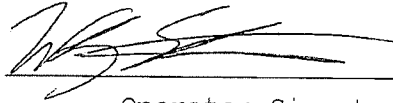

Operator Signature
WESLEY STRAIGHT

Remarks:

Form 106-I8000