

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006674

Location: BMCD

BURLEIGH MORTON  
DETENTION CENTER

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 521 psi Regulator: 510 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 34917080A3 Cyl #: 25 Exp. Date: 2.5.20

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.080</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.082</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: REPLACED BREATH HOSE O-RING, -  
REPLACED SIMULATOR RETURN O-RING, ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. [Signature]  
Breath Analyst Signature

4.11.19  
Date

NA  
Reviewed by

NA  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer             Alcohol Analyzer  
North Dakota Model 8000             SN 80-006674  
Location = BMCD                     8164.14.00 09/16  
04/11/2019                             15:27

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=|  
ABCDEFGHIJKLMNopqrstuvWxyz!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|  
ABCDEFGHIJKLMNopqrstuvWxyz!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:                    Enabled  
Start Test Sequence:                 DABACABA  
Display Prelim Rslt?                 Yes  
Display Third Digit?                 Yes  
Inhib Printer(Y/N)?                 No  
Display Volume?                     No  
Disable On Memfull?                 Yes  
# of Print Copies?                   1  
Select Std (D/W/I)?                 Dry  
Standard Value?                     0.080  
Standard Lot #?                     34917080A3  
Standard Cyl #?                     25  
Standard Expiration?                 02/05/2020  
Oper No?                             666666

Flow Cal. Date:                     06/28/2017  
Slope                                 707  
Intercept                             -708478

IR Calibration Date:                 06/28/2017  
                                       3um                     9um

	3um	9um
0th Coef(*100):	-3915	-18046
1st Coef(*100):	256520	127506
2nd Coef(*100):	3843	1644
H2O adj(mg/l*10k):	428	568

\*\*\*\*\* Printer Test End \*\*\*\*\*

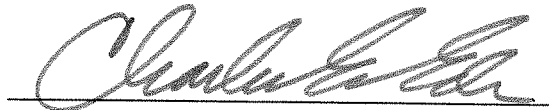

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:36
02 Std. Sol.	0.020	15:36
03 Room Air	0.000	15:37
04 Std. Sol.	0.021	15:38
05 Room Air	0.000	15:38
06 Std. Sol.	0.020	15:39
07 Room Air	0.000	15:39

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

Low AC  
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:42

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Sol.	0.294	15:43
03 Room Air	0.000	15:44
04 Std. Sol.	0.298	15:44
05 Room Air	0.000	15:45
06 Std. Sol.	0.298	15:46
07 Room Air	0.000	15:46


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 08

Oper No. = 666666

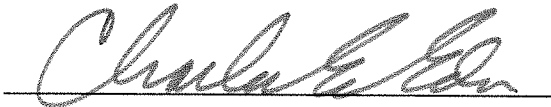

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:56

Test	AC	Time
01 Room Air	0.000	15:57
02 *Subject Test	INT*	15:57
03 Room Air	0.000	15:58

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS821456  
Lot No = 34917080A3  
Cyl No = 25  
Expiration Date = 02/05/2020  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10 AC + 0.05% ACETONE*  
Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:53

Test	AC	Time
01 Diagnostic	OK	15:54
02 Room Air	0.000	15:54
03 *Subject Test	RFI*	15:54
04 Room Air	0.000	15:55

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = RFI CHECK  
Dr. Lic. = ND/DIS821456  
Lot No = 34917080A3  
Cyl No = 25  
Expiration Date = 02/05/2020  
County = 08      Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:49
02 Std. Gas	0.081	15:49
03 Room Air	0.000	15:50
04 Std. Gas	0.080	15:50
05 Room Air	0.000	15:51
06 Std. Gas	0.081	15:51
07 Room Air	0.000	15:52

Lot No = 34917080A3  
Cyl No = 25  
Exp Date = 02/05/2020  
County = 08      Oper No. = 666666

  
Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      15:58

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:58
02 Std. Gas	0.081	15:59
03 Room Air	0.000	15:59
04 Std. Gas	0.081	16:00
05 Room Air	0.000	16:00
06 Std. Gas	0.081	16:01
07 Room Air	0.000	16:01

Lot No = 34917080A3  
Cyl No = 25  
Exp Date = 02/05/2020  
County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080AC*

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006674  
Location = BMCD      8164.14.00 09/16  
04/11/2019      16:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:02
02 Std. Gas	0.081	16:02
03 Room Air	0.000	16:03
04 Std. Gas	0.081	16:03
05 Room Air	0.000	16:04
06 Std. Gas	0.082	16:04
07 Room Air	0.000	16:05

Lot No = 34917080A3  
Cyl No = 25  
Exp Date = 02/05/2020  
County = 08

Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000