

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006667

Location: GOLDEN VALLEY
S.O. (GLDN)

A. General Setup and Checks:

- 1. Diagnostic Tests Pass and Instrument in Ready Mode
- 2. Breath tube heated
- 3. Date, time and location code (Level 2,E). Re-set if necessary.
- 4. Print test (Level 1,P). Sign and attach test record.
- 5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 983 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6.8.19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

- 2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS5 Exp. Date: NA
 Display reads "Interferent Detect"
- 3. RFI Check (CMS Mode) Display reads "RFI Detect"
- 4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34917080A3 Cyl #: 8 Exp. Date: 2.5.20

| | | | | | |
|---------|--------------|--------|--------------|--------|--------------|
| Test 1 | <u>0.080</u> | Test 4 | <u>0.081</u> | Test 7 | <u>0.081</u> |
| Test 2 | <u>0.080</u> | Test 5 | <u>0.081</u> | Test 8 | <u>0.081</u> |
| Test 3 | <u>0.081</u> | Test 6 | <u>0.081</u> | Test 9 | <u>0.081</u> |
| Average | <u>0.081</u> | | | | |

C. Remarks/Maintenance: REPLACED BREATH HOSE O-RING,
ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Edwards
Breath Analyst Signature
NA

4.12.19
Date
NA

Reviewed by _____

Date _____

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:30

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 08:31 |
| 02 Std. Sol. | 0.019 | 08:31 |
| 03 Room Air | 0.000 | 08:32 |
| 04 Std. Sol. | 0.019 | 08:33 |
| 05 Room Air | 0.000 | 08:33 |
| 06 Std. Sol. | 0.019 | 08:34 |
| 07 Room Air | 0.000 | 08:34 |

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 17

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:35

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 08:35 |
| 02 Std. Sol. | 0.297 | 08:36 |
| 03 Room Air | 0.000 | 08:37 |
| 04 Std. Sol. | 0.297 | 08:37 |
| 05 Room Air | 0.000 | 08:38 |
| 06 Std. Sol. | 0.299 | 08:39 |
| 07 Room Air | 0.000 | 08:39 |


08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 17

Oper No. = 666666

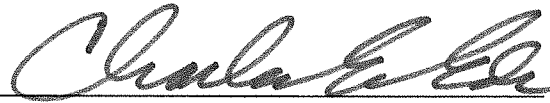

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:19

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 08:20 |
| 02 *Subject Test | INT* | 08:20 |
| 03 Room Air | 0.000 | 08:21 |

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 8
Expiration Date = 02/05/2020
County = 17 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:22

| Test | AC | Time |
|------------------|-------|-------|
| 01 Diagnostic | OK | 08:23 |
| 02 Room Air | 0.000 | 08:23 |
| 03 *Subject Test | RFI* | 08:23 |
| 04 Room Air | 0.000 | 08:24 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34917080A3
Cyl No = 8
Expiration Date = 02/05/2020
County = 17 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:15

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 08:16 |
| 02 Std. Gas | 0.080 | 08:16 |
| 03 Room Air | 0.000 | 08:17 |
| 04 Std. Gas | 0.080 | 08:17 |
| 05 Room Air | 0.000 | 08:18 |
| 06 Std. Gas | 0.081 | 08:18 |
| 07 Room Air | 0.000 | 08:19 |

Lot No = 34917080A3
Cyl No = 8
Exp Date = 02/05/2020
County = 17 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:40

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 08:40 |
| 02 Std. Gas | 0.081 | 08:41 |
| 03 Room Air | 0.000 | 08:41 |
| 04 Std. Gas | 0.081 | 08:42 |
| 05 Room Air | 0.000 | 08:42 |
| 06 Std. Gas | 0.081 | 08:43 |
| 07 Room Air | 0.000 | 08:43 |

Lot No = 34917080A3
Cyl No = 8
Exp Date = 02/05/2020
County = 17 Oper No. = 666666

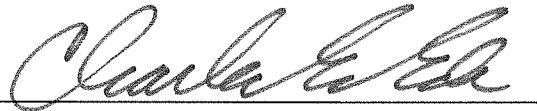

Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006667
Location = GLDN 8164.14.00 09/16
04/12/2019 08:43

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 08:44 |
| 02 Std. Gas | 0.081 | 08:44 |
| 03 Room Air | 0.000 | 08:45 |
| 04 Std. Gas | 0.081 | 08:45 |
| 05 Room Air | 0.000 | 08:45 |
| 06 Std. Gas | 0.081 | 08:46 |
| 07 Room Air | 0.000 | 08:46 |

Lot No = 34917080A3
Cyl No = 8
Exp Date = 02/05/2020
County = 17

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000