

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006688

Location: KIDDER COUNTY
S.O. (STEE)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 996 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR3375 Lot #: 17160 Exp. Date: 6-8-19
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10-11-19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: JCS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 60 Exp. Date: 2-5-21

Test 1	<u>0.082</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.082</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.081</u>	Test 6	<u>0.082</u>	Test 9	<u>0.081</u>
Average <u>0.082</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr
Breath Analyst Signature

NA

5-7-2019
Date

NA
Date

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 16:06

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:06
02 Std. Sol.	0.018	16:07
03 Room Air	0.000	16:07
04 Std. Sol.	0.020	16:08
05 Room Air	0.000	16:09
06 Std. Sol.	0.020	16:09
07 Room Air	0.000	16:10

08 Sim Temp = 34.0°C

Simul Ser No = DR3375

Std Sol No = 17160

County = 22

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

LOW AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 16:11

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:11
02 Std. Sol.	0.298	16:12
03 Room Air	0.000	16:13
04 Std. Sol.	0.300	16:14
05 Room Air	0.000	16:14
06 Std. Sol.	0.302	16:15
07 Room Air	0.000	16:15

08 Sim Temp = 34.0°C

Simul Ser No = DR7111

Std Sol No = 17350

County = 22

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 15:39

Test	AC	Time
01 Room Air	0.000	15:40
02 *Subject Test	INT*	15:40
03 Room Air	0.000	15:41

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 34418080A2
Cyl No = 60
Expiration Date = 02/05/2021
County = 22 Oper No. = 666666

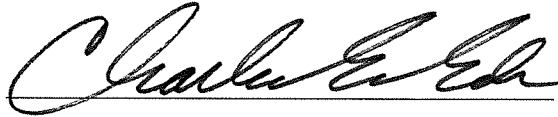

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 15:46

Test	AC	Time
01 Diagnostic	OK	15:47
02 Room Air	0.000	15:47
03 *Subject Test	RFI*	15:47
04 Room Air	0.000	15:48

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 34418080A2
Cyl No = 60
Expiration Date = 02/05/2021
County = 22 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: **RFI CHECK**

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 15:41

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:42
02 Std. Gas	0.082	15:42
03 Room Air	0.000	15:42
04 Std. Gas	0.082	15:43
05 Room Air	0.000	15:43
06 Std. Gas	0.081	15:44
07 Room Air	0.000	15:44

Lot No = 34418080A2
Cyl No = 60
Exp Date = 02/05/2021
County = 22 Oper No. = 666666

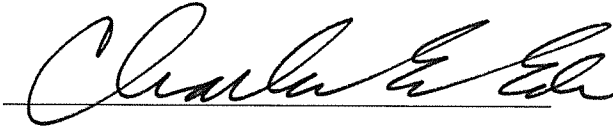

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 16:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:01
02 Std. Gas	0.082	16:01
03 Room Air	0.000	16:02
04 Std. Gas	0.082	16:02
05 Room Air	0.000	16:03
06 Std. Gas	0.082	16:03
07 Room Air	0.000	16:03

Lot No = 34418080A2
Cyl No = 60
Exp Date = 02/05/2021
County = 22 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
05/07/2019 16:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:17
02 Std. Gas	0.082	16:17
03 Room Air	0.000	16:18
04 Std. Gas	0.082	16:18
05 Room Air	0.000	16:18
06 Std. Gas	0.081	16:19
07 Room Air	0.000	16:19

Lot No = 34418080A2
Cyl No = 60
Exp Date = 02/05/2021
County = 22 Oper No. = 666666

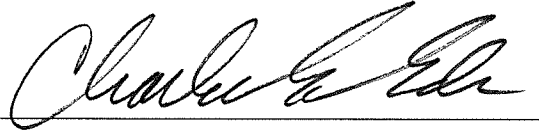

Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000