Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer[®] 8000 Serial Number: 80-006671

Location: GRIGGS County 5.0. (COOP)

- A. General Setup and Checks:
 - 1. Diagnostic Tests Pass and Instrument in Ready Mode
 - 2. Breath tube heated
 - 3. X Date, time and location code (Level 2,E). Re-set if necessary.
 - 4. X Print test (Level 1,P). Sign and attach test record.
 - 5. X Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 85 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

X Configure simulator for the following tests (Level 1,S).

- 1. Wet Calibration Check
 - a. X Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC Sim. Ser #: DR5/31 Lot #: 20/8/00 Exp. Date: 10.24.20
 - b. X High AC (Level 1,C): Known Value ≥ 0.25 AC: <u>0.300</u> AC Sim. Ser #: <u>DR7/11</u> Lot #: <u>17350</u> Exp. Date: <u>10.11.19</u>

Configure dry gas standard for the following tests (Level 1,S).

- 2. X Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone Sim. Ser #: DR 7081 Lot #: ICS 5 Exp. Date: NA
 X Display reads "Interferent Detect"
- 3. XRFI Check (CMS Mode) X Display reads "RFI Detect"
- C. Remarks/Maintenance: ANNUAL INSPECTION

X Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

5.14.19 Date

02/18 Issuing Authority - CEE

Page 1 of 1

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006671 Location = COOP 8164.14.00 09/16 05/14/2019 13:22 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? Standard Cyl #? 13518080A6 025 Standard Expiration? 08/05/2020 Oper No? 666666 Flow Cal. Date: 08/22/2016 Slope 703 Intercept -691868 IR Calibration Date: 08/22/2016 3um 9um _____ Oth Coef(*100): -17135 -24345 1st Coef(*100): 254798 2nd Coef(*100): 3386 132856 1354 H2O adj(mg/l*10k): 610 539 ********** Printer Test End **********

Operator Signature CHARLES EDER

IEST RINT

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006671Location = COOP8164.14.00 09/1605/14/201913:36

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.018 0.000 0.018 0.000 0.018 0.000	13:36 13:37 13:37 13:38 13:39 13:39 13:40

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR5131 Std Sol No = 201810D County = 20

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC 0.070AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000Alcohol Analyzer
SN 80-006671Location = COOP8164.14.00 09/1605/14/201913:42

	WET CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Sol. 03 Room Air 04 Std. Sol. 05 Room Air 06 Std. Sol. 07 Room Air	0.000 0.293 0.000 0.294 0.000 0.296 0.000	13:42 13:43 13:44 13:44 13:45 13:46

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = DR7111 Std Sol No = 17350 County = 20

Oper No. = 666666

Operator Signature CHARLES EDER

H164 AC

0.300 AC

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006671 Location = COOP8164.14.00 09/16 05/14/2019 13:32 Test AC Time 01 Room Air 0.000 13:33 02 *Subject Test INT* 13:33 03 Room Air 0.000 13:34 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 01/01/1982Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS821456Lot No = 13518080A6Cyl No = 025Expiration Date = 08/05/2020County = 20Oper No. = 666666

Operator Signature CHARLES EDER

Interferent Check 0.WAC + 0.05% ACETONE

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006671 Location = COOP8164.14.00 09/16 05/14/2019 13:26 Test AC Time 01 Diagnostic OK 13:27 02 Room Air 0.000 13:28 03 *Subject Test RFI* 13:28 04 Room Air 0.000 13:28 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 01/01/1982Sub Sex = Female Weight = 150Test = DUI Cit = RFI CHECK Dr. Lic. = ND/DIS821456Lot No = 13518080A6Cyl No = 025Expiration Date = 08/05/2020County = 20Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

UM.

Operator Signature CHARLES EDER

Remarks:

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alc North Dakota Model 8000 Alcohol Analyzer SN 80-006671 Location = COOP8164.14.00 09/16 05/14/2019 13:23

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.081 0.000 0.081 0.000	13:23 13:24 13:24 13:25 13:25 13:25 13:26 13:26

Lot No = 13518080A6Cyl No = 025Exp Date = 08/05/2020County = 20

Oper No. = 666666

Remarks:

Operator Signature CHARLES EDER CALIBRATION CUECK 0.080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006671 Location = COOP8164.14.00 09/16 05/14/2019 13:29

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.082 0.000 0.081 0.000 0.081 0.000	13:29 13:29 13:30 13:30 13:31 13:31 13:32

Lot No = 13518080A6Cyl No = 025Exp Date = 08/05/2020County = 20

Oper No. = 666666

ĹŰ

Operator Signature

ALIBRATION CHECK 0.080 AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006671 Location = COOP8164.14.00 09/16 05/14/2019 13:48

DRY CAL CHECK Test AC Time 01 Room Air 0.000 13:49 02 Std. Gas 0.081 13:49 03 Room Air 0.000 13:49 04 Std. Gas 0.082 13:50 05 Room Air 0.000 13:50 06 Std. Gas 0.082 13:51 07 Room Air 0.000 13:51

Lot No = 13518080A6Cyl No = 25Exp Date = 08/05/2020County = 20

Oper No. = 666666

CHARLES EDER ALIBRATION CHECK

0.080 AC

Form 106-18000