

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006684

Location: WEST FARGO
P.D. (WFPD)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 878 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5131 Lot #: 201810D Exp. Date: 10.24.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: FC5 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 4 Exp. Date: 9.5.19

Test 1	<u>0.081</u>	Test 4	<u>0.079</u>	Test 7	<u>0.080</u>
Test 2	<u>0.081</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.081</u>	Test 6	<u>0.080</u>	Test 9	<u>0.081</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, EXCHANGED
BROTHER LASER PRINTER WITH LEXMARK LASER
PRINTER (EXTERNAL PRINTER)

Instrument is acceptable to be used in the field.

Charles E. Eden
Breath Analyst Signature

5.14.19
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 17:56

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:56
02 Std. Sol.	0.018	17:57
03 Room Air	0.000	17:58
04 Std. Sol.	0.018	17:58
05 Room Air	0.000	17:59
06 Std. Sol.	0.018	17:59
07 Room Air	0.000	18:00

08 Sim Temp = 34.0°C

Simul Ser No = DR5131
Std Sol No = 201810D
County = 09 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 18:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:17
02 Std. Sol.	0.297	18:18
03 Room Air	0.000	18:19
04 Std. Sol.	0.297	18:19
05 Room Air	0.000	18:20
06 Std. Sol.	0.298	18:21
07 Room Air	0.000	18:21

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 17350
County = 09 Oper No. = 666666

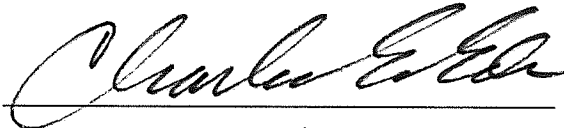

Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 17:41

Test	AC	Time
01 Room Air	0.000	17:42
02 *Subject Test	INT*	17:43
03 Room Air	0.000	17:43

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 4
Expiration Date = 09/05/2019
County = 09 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 17:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:32
02 Std. Gas	0.081	17:33
03 Room Air	0.000	17:33
04 Std. Gas	0.081	17:34
05 Room Air	0.000	17:34
06 Std. Gas	0.081	17:35
07 Room Air	0.000	17:35

Lot No = 19817080A4
Cyl No = 4
Exp Date = 09/05/2019
County = 09 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 17:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:52
02 Std. Gas	0.079	17:53
03 Room Air	0.000	17:53
04 Std. Gas	0.080	17:54
05 Room Air	0.000	17:54
06 Std. Gas	0.080	17:54
07 Room Air	0.000	17:55

Lot No = 19817080A4
Cyl No = 4
Exp Date = 09/05/2019
County = 09 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = WFPD 8164.14.00 09/16
05/14/2019 18:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:23
02 Std. Gas	0.080	18:23
03 Room Air	0.000	18:24
04 Std. Gas	0.080	18:24
05 Room Air	0.000	18:25
06 Std. Gas	0.081	18:25
07 Room Air	0.000	18:26

Lot No = 19817080A4
Cyl No = 4
Exp Date = 09/05/2019
County = 09 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000