

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006508

Location: WAHPETON LEC.
(WAHP)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 660 psi Regulator: 650 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5131 Lot #: 201810D Exp. Date: 10.24.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 1981708044 Cyl #: 24 Exp. Date: 9.5.19

Test 1	<u>0.079</u>	Test 4	<u>0.080</u>	Test 7	<u>0.080</u>
Test 2	<u>0.079</u>	Test 5	<u>0.080</u>	Test 8	<u>0.080</u>
Test 3	<u>0.080</u>	Test 6	<u>0.081</u>	Test 9	<u>0.079</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Eds
Breath Analyst Signature

5.15.19
Date

NA

NA

Reviewed by

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 13:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMN O PQRSTUVWXYZ!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxy1234567890-=
|
ABCDEFGHIJKLMN O PQRSTUVWXYZ!@#%&^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 24
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 08/20/2015
Slope 644
Intercept -568181

IR Calibration Date: 08/20/2015
 3um 9um

0th Coef(*100): -22365 -18627
1st Coef(*100): 256969 136095
2nd Coef(*100): 3168 1444
H2O adj(mg/l*10k): 693 453

***** Printer Test End *****

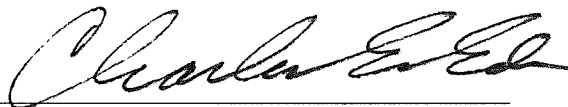

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 13:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:53
02 Std. Sol.	0.019	13:54
03 Room Air	0.000	13:54
04 Std. Sol.	0.019	13:55
05 Room Air	0.000	13:56
06 Std. Sol.	0.019	13:56
07 Room Air	0.000	13:57

08 Sim Temp = 34.0°C

Simul Ser No = DR5131
Std Sol No = 201810D
County = 39 Oper No. = 666666

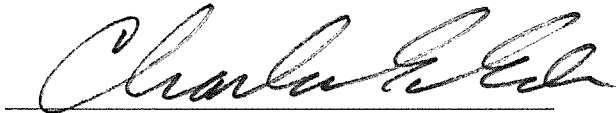

Operator Signature
CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 13:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:58
02 Std. Sol.	0.297	13:59
03 Room Air	0.000	14:00
04 Std. Sol.	0.298	14:00
05 Room Air	0.000	14:01
06 Std. Sol.	0.298	14:02
07 Room Air	0.000	14:02

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 17350
County = 39 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 14:06

Test	AC	Time
01 Room Air	0.000	14:07
02 *Subject Test	INT*	14:07
03 Room Air	0.000	14:08

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 24
Expiration Date = 09/05/2019
County = 39 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 14:09

Test	AC	Time
01 Diagnostic	OK	14:10
02 Room Air	0.000	14:10
03 *Subject Test	RFI*	14:10
04 Room Air	0.000	14:11

*Invalid Test
 Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 24
Expiration Date = 09/05/2019
County = 39 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 14:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:11
02 Std. Gas	0.079	14:12
03 Room Air	0.000	14:12
04 Std. Gas	0.079	14:13
05 Room Air	0.000	14:13
06 Std. Gas	0.080	14:14
07 Room Air	0.000	14:14

Lot No = 19817080A4
Cyl No = 24
Exp Date = 09/05/2019
County = 39 Oper No. = 666666

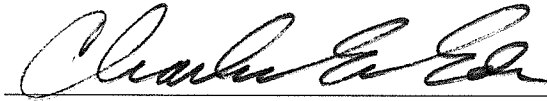

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 14:14

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:15
02 Std. Gas	0.080	14:15
03 Room Air	0.000	14:16
04 Std. Gas	0.080	14:16
05 Room Air	0.000	14:17
06 Std. Gas	0.081	14:17
07 Room Air	0.000	14:18

Lot No = 19817080A4
Cyl No = 24
Exp Date = 09/05/2019
County = 39 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006508
Location = WAHP 8164.13.00 06/09
05/15/2019 14:18

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:18
02 Std. Gas	0.080	14:19
03 Room Air	0.000	14:19
04 Std. Gas	0.080	14:20
05 Room Air	0.000	14:20
06 Std. Gas	0.079	14:21
07 Room Air	0.000	14:21

Lot No = 19817080A4
Cyl No = 24
Exp Date = 09/05/2019
County = 39 Oper No. = 666666

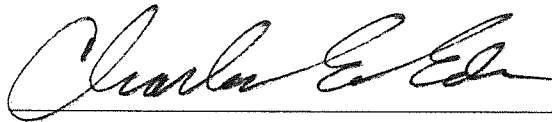

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000