

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006683

Location: OAKES P.D.
(OKPD)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 900 psi Regulator: 875 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: DR5131 Lot #: 201810D Exp. Date: 10.24.20
 - b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 13 Exp. Date: 9.5.19

Test 1	<u>0.080</u>	Test 4	<u>0.080</u>	Test 7	<u>0.079</u>
Test 2	<u>0.079</u>	Test 5	<u>0.079</u>	Test 8	<u>0.080</u>
Test 3	<u>0.079</u>	Test 6	<u>0.080</u>	Test 9	<u>0.079</u>
Average	<u>0.079</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Edin
Breath Analyst Signature

5.16.2019
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:18

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 13
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 694
Intercept -595721

IR Calibration Date: 09/06/2016

	3um	9um
0th Coef(*100):	-8889	-21482
1st Coef(*100):	254010	129695
2nd Coef(*100):	3922	1447
H2O adj(mg/l*10k):	608	570

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:32
02 Std. Sol.	0.018	10:32
03 Room Air	0.000	10:33
04 Std. Sol.	0.019	10:34
05 Room Air	0.000	10:34
06 Std. Sol.	0.019	10:35
07 Room Air	0.000	10:35

08 Sim Temp = 34.0°C

Simul Ser No = DR5131
Std Sol No = 201810D
County = 11 Oper No. = 666666

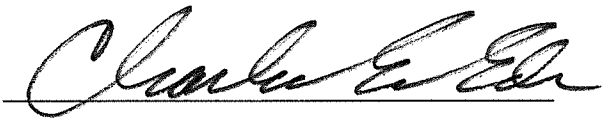

Operator Signature
CHARLES EDER

Remarks:

*Low AC
0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:44
02 Std. Sol.	0.295	10:45
03 Room Air	0.000	10:45
04 Std. Sol.	0.296	10:46
05 Room Air	0.000	10:46
06 Std. Sol.	0.296	10:47
07 Room Air	0.000	10:48

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 17350
County = 11 Oper No. = 666666

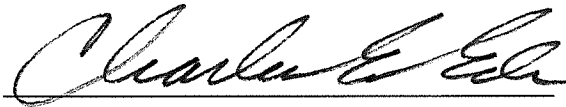

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:23

Test	AC	Time
01 Room Air	0.000	10:23
02 *Subject Test	INT*	10:24
03 Room Air	0.000	10:25

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 13
Expiration Date = 09/05/2019
County = 11 Oper No. = 666666

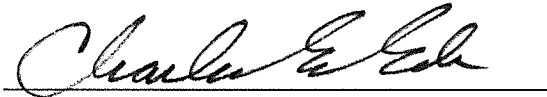

Operator Signature
CHARLES EDER

Remarks: *Interferent Check*
0.10 AC + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:25

Test	AC	Time
01 Diagnostic	OK	10:26
02 Room Air	0.000	10:26
03 *Subject Test	RFI*	10:26
04 Room Air	0.000	10:27

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 13
Expiration Date = 09/05/2019
County = 11 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: **RFI CHECK**

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:20
02 Std. Gas	0.080	10:20
03 Room Air	0.000	10:20
04 Std. Gas	0.079	10:21
05 Room Air	0.000	10:21
06 Std. Gas	0.079	10:22
07 Room Air	0.000	10:22

Lot No = 19817080A4
Cyl No = 13
Exp Date = 09/05/2019
County = 11 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:27

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Gas	0.080	10:28
03 Room Air	0.000	10:28
04 Std. Gas	0.079	10:29
05 Room Air	0.000	10:29
06 Std. Gas	0.080	10:30
07 Room Air	0.000	10:30

Lot No = 19817080A4
Cyl No = 13
Exp Date = 09/05/2019
County = 11 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECK*
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = OKPD 8164.14.00 09/16
05/16/2019 10:48

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:49
02 Std. Gas	0.079	10:49
03 Room Air	0.000	10:50
04 Std. Gas	0.080	10:50
05 Room Air	0.000	10:51
06 Std. Gas	0.079	10:51
07 Room Air	0.000	10:51

Lot No = 19817080A4
Cyl No = 13
Exp Date = 09/05/2019
County = 11 Oper No. = 666666

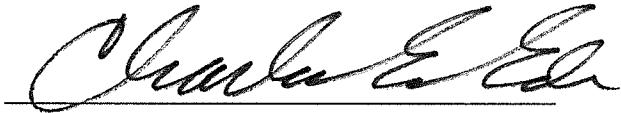

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000