

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006680

Location: LAMOURE COUNTY
S.O. (LAMO)

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 919 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5131 Lot #: 201810D Exp. Date: 10.24.20
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7111 Lot #: 17350 Exp. Date: 10.11.19

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR7089 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 19817080A4 Cyl #: 7 Exp. Date: 9.5.19

Test 1	<u>0.081</u>	Test 4	<u>0.080</u>	Test 7	<u>0.078</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.079</u>
Test 3	<u>0.080</u>	Test 6	<u>0.079</u>	Test 9	<u>0.079</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION,
REPLACED EXTERNAL LASER PRINTER

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

Date

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = LAMO 8164.14.00 09/16
05/16/2019 08:44

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 19817080A4
Standard Cyl #? 7
Standard Expiration? 09/05/2019
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 681
Intercept -620591

IR Calibration Date: 09/06/2016
 3um 9um

0th Coef(*100): -14265 -21271
1st Coef(*100): 260743 129687
2nd Coef(*100): 3906 1392
H2O adj(mg/l*10k): 742 684

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = LAMO 8164.14.00 09/16
05/16/2019 09:05

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:06
02 Std. Sol.	0.018	09:07
03 Room Air	0.000	09:07
04 Std. Sol.	0.018	09:08
05 Room Air	0.000	09:09
06 Std. Sol.	0.018	09:09
07 Room Air	0.000	09:10

08 Sim Temp = 34.0°C

Simul Ser No = DR5131

Std Sol No = 201810D

County = 23

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

Low AC
0.020AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = LAMO 8164.14.00 09/16
05/16/2019 09:15

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:16
02 Std. Sol.	0.307	09:17
03 Room Air	0.000	09:17
04 Std. Sol.	0.307	09:18
05 Room Air	0.000	09:18
06 Std. Sol.	0.306	09:19
07 Room Air	0.000	09:20

08 Sim Temp = 34.0°C

Simul Ser No = DR7111
Std Sol No = 17350
County = 23 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = LAMO 8164.14.00 09/16
05/16/2019 08:59

Test	AC	Time
01 Room Air	0.000	09:00
02 *Subject Test	INT*	09:01
03 Room Air	0.000	09:01

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS821456
Lot No = 19817080A4
Cyl No = 7
Expiration Date = 09/05/2019
County = 23 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
CEE
0.10 AC + ~~0.08~~ 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = LAMO 8164.14.00 09/16
05/16/2019 09:20

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:21
02 Std. Gas	0.078	09:21
03 Room Air	0.000	09:22
04 Std. Gas	0.079	09:22
05 Room Air	0.000	09:23
06 Std. Gas	0.079	09:23
07 Room Air	0.000	09:24

Lot No = 19817080A4
Cyl No = 7
Exp Date = 09/05/2019
County = 23

Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000