



ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM

SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>CHARLES E. EDER, STATE TOX., ANNUAL INSPECTION</i>		
Location <i>KENMARE P.D. (KMA)</i>	Intoxilyzer® Serial Number <i>80-006691</i>	
Gas Lot Number <i>34418080A2</i>	Gas Cylinder Number <i>49</i>	Gas Expiration Date <i>2/5/2021</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. *0.080* AC

B. *0.081* AC

C. *0.081* AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency. *NEW LOCATION*

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>Charles E. Eder</i>	Date <i>5/30/19</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eder</i>	Date <i>5/30/19</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006691
Location = KMPD 8164.14.00 09/16
05/30/2019 11:46

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:47
02 Std. Gas	0.080	11:47
03 Room Air	0.000	11:48
04 Std. Gas	0.081	11:48
05 Room Air	0.000	11:49
06 Std. Gas	0.081	11:49
07 Room Air	0.000	11:50

Lot No = 34418080A2
Cyl No = 49
Exp Date = 02/05/2021
County = 51 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *NEW GAS CYLINDER*

Form 106-I8000