

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006513

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 457 psi Regulator: 475 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR337B Lot #: 201712B Exp. Date: 12.12.19
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: DR7350 Lot #: 201803H Exp. Date: 3.22.20

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 5 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34418080A2 Cyl #: 45 Exp. Date: 2.5.21

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.082</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.082</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.082</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED & TRAINED EXTERNAL BATTERY. (6.12.19)

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

6/14/2019
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:38

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:38
02 Std. Sol.	0.019	13:39
03 Room Air	0.000	13:39
04 Std. Sol.	0.019	13:40
05 Room Air	0.000	13:41
06 Std. Sol.	0.019	13:41
07 Room Air	0.000	13:42

08 Sim Temp = 34.0°C

Simul Ser No = DR3378
Std Sol No = 201712B
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:43

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:43
02 Std. Sol.	0.297	13:44
03 Room Air	0.000	13:45
04 Std. Sol.	0.297	13:46
05 Room Air	0.000	13:46
06 Std. Sol.	0.297	13:47
07 Room Air	0.000	13:47

08 Sim Temp = 34.0°C

Simul Ser No = DR7350
Std Sol No = 201803H
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

HIGH AC
CEE D.O. 0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:51

Test	AC	Time
01 Room Air	0.000	13:51
02 *Subject Test	INT*	13:52
03 Room Air	0.000	13:52

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:53

Test	AC	Time
01 Diagnostic	OK	13:54
02 Room Air	0.000	13:54
03 *Subject Test	RFI*	13:54
04 Room Air	0.000	13:55

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 34418080A2
Cyl No = 45
Expiration Date = 02/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:



Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:56
02 Std. Gas	0.081	13:56
03 Room Air	0.000	13:56
04 Std. Gas	0.081	13:57
05 Room Air	0.000	13:57
06 Std. Gas	0.081	13:58
07 Room Air	0.000	13:58

Lot No = 34418080A2
Cyl No = 45
Exp Date = 02/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 13:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:00
02 Std. Gas	0.081	14:00
03 Room Air	0.000	14:00
04 Std. Gas	0.081	14:01
05 Room Air	0.000	14:01
06 Std. Gas	0.081	14:02
07 Room Air	0.000	14:02

Lot No = 34418080A2
Cyl No = 45
Exp Date = 02/05/2021
County = 08 Oper No. = 666666

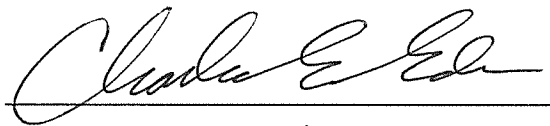

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = TOXL 8164.13.00 06/09
06/14/2019 14:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:03
02 Std. Gas	0.082	14:04
03 Room Air	0.000	14:04
04 Std. Gas	0.082	14:04
05 Room Air	0.000	14:05
06 Std. Gas	0.082	14:05
07 Room Air	0.000	14:06

Lot No = 34418080A2
Cyl No = 45
Exp Date = 02/05/2021
County = 08 Oper No. = 666666

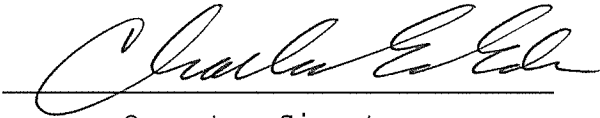

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000