



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>Jordan Buffett</i>		
Location <i>Richland County Jail</i>	Intoxilyzer® Serial Number <i>80-006508</i>	
Gas Lot Number <i>34418080AZ</i>	Gas Cylinder Number <i>8</i>	Gas Expiration Date <i>02-05-2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.080* AC  
 B. *0.080* AC  
 C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>Jordan Buffett</i>	Date <i>08-31-2019</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Ede</i>	Date <i>09-03-2019</i>

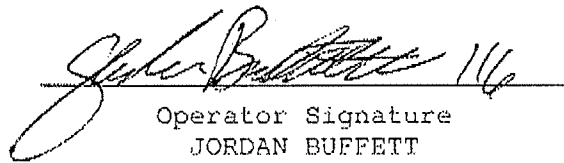
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006508  
Location = WAHP      8164.13.00 06/09  
08/31/2019      14:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:10
02 Std. Gas	0.080	14:10
03 Room Air	0.000	14:11
04 Std. Gas	0.080	14:11
05 Room Air	0.000	14:12
06 Std. Gas	0.081	14:12
07 Room Air	0.000	14:13

Lot No = 34418080A2  
Cyl No = 8  
Exp Date = 02/05/2021  
County = 39      Oper No. = 131718

  
Operator Signature  
JORDAN BUFFETT

Remarks: *Good ACA*

Form 106-1B000