



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Lucas Kuntz</i>		
Location <i>Wishak Police Department</i>	Intoxilyzer® Serial Number <i>80-007087</i>	
Gas Lot Number <i>34418080A2</i>	Gas Cylinder Number <i>012</i>	Gas Expiration Date <i>2-5-2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. *0.080* AC
 B. *0.080* AC
 C. *0.079* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>11-12-19</i>
Reviewed By (Crime Laboratory Use Only) <i>1664 [Signature]</i>	Date <i>11.15.2019</i>

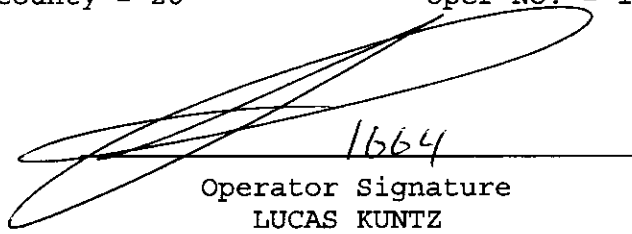
Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-007087
Location = WISH 8164.16.00 09/18
11/12/2019 13:03

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:03
02 Std. Gas	0.080	13:03
03 Room Air	0.000	13:04
04 Std. Gas	0.080	13:04
05 Room Air	0.000	13:05
06 Std. Gas	0.079	13:05
07 Room Air	0.000	13:06

Lot No = 34418080A2
Cyl No = 12
Exp Date = 02/05/2021
County = 26 Oper No. = 131099


Operator Signature
LUCAS KUNTZ

Remarks:

Form 106-I8000