



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tim Bleth</i>		
Location <i>BSPD</i>	Intoxilyzer® Serial Number <i>80-006675</i>	
Gas Lot Number <i>24119080A1</i>	Gas Cylinder Number <i>31</i>	Gas Expiration Date <i>11-5-2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C). *CEE 1-6-20 REC'D ACA TEST*
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0. *082* AC
 B. 0. *081* AC
 C. 0. *082* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory: *REC'D PAPERWORK*
CEE 1-6-20
 A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 B. ACA Test Record.

Chemical Test Operator Signature <i>Tim Bleth</i>	Date <i>1-6-2020</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edr</i>	Date <i>1-6-2020</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006675
Location = BSPD 8164.14.00 09/16
01/06/2020 11:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:50
02 Std. Gas	0.082	11:51
03 Room Air	0.000	11:51
04 Std. Gas	0.081	11:51
05 Room Air	0.000	11:52
06 Std. Gas	0.082	11:52
07 Room Air	0.000	11:53

Lot No = 24119080A1
Cyl No = 31
Exp Date = 11/05/2021
County = 08 Oper No. = 069823



Operator Signature
TIMOTHY BLETH

Remarks: *ACA - cylinder change*

Form 106-I8000