



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Tom Sharp</i>		
Location <i>New Salem</i>	Intoxilyzer® Serial Number <i>80-006676</i>	
Gas Lot Number <i>24119080a1</i>	Gas Cylinder Number <i>16</i>	Gas Expiration Date <i>11/5/2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C). *RECEIVED ACA TEST*  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
*CEE*  
*2-13-20*  
 A. 0.083 AC  
 B. 0.082 AC  
 C. 0.082 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>[Signature]</i>	Date <i>2/10/2020</i>
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date <i>2/13/20</i>

Intoxilyzer Test Record and Checklist  
NDDAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006676  
Location = NS      8164.14.00 09/16  
02/10/2020      10:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:27
02 Std. Gas	0.083	10:28
03 Room Air	0.000	10:28
04 Std. Gas	0.082	10:28
05 Room Air	0.000	10:29
06 Std. Gas	0.082	10:29
07 Room Air	0.000	10:30

Lot No = 24119080A1  
Cyl No = 16  
Exp Date = 11/05/2021  
County = 30      Oper No. = 920611

  
Operator Signature  
TOM SHARP

Remarks:

Form 106-18000