



**ETHANOL GAS STANDARD CYLINDER REPORT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Thomas, Kelly, Jo</i>		
Location <i>Carrington Police Department</i>	Intoxilyzer® Serial Number <i>80-006512</i>	
Gas Lot Number <i>24119080A1</i>	Gas Cylinder Number <i>13</i>	Gas Expiration Date <i>11-05-20</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).  
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).  
 A. *0.019* AC  
 B. *0.019* AC  
 C. *0.019* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
  - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
  - B. ACA Test Record.

Chemical Test Operator Signature <i>JKW 4/28/20</i>	Date <i>04-22-20</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eden</i>	Date <i>4.30.20</i>

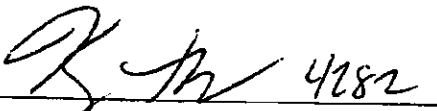
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006512  
Location = CARR      8164.14.00 09/16  
04/22/2020      18:16

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:17
02 Std. Gas	0.079	18:18
03 Room Air	0.000	18:18
04 Std. Gas	0.079	18:18
05 Room Air	0.000	18:19
06 Std. Gas	0.079	18:19
07 Room Air	0.000	18:20

Lot No = 24119080A1  
Cyl No = 13  
Exp Date = 11/05/2021  
County = 16      Oper No. = 132640

  
Operator Signature  
KELLY THOMAS

Remarks:

Form 106-I8000