



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number <i>80-006685</i>	Instrument Location <i>GMH-Carrison</i>
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *None*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>5/7/2020</i>
---	-------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>[Signature]</i>	Certified Date <i>6-18-20</i>
---	----------------------------------

**NOTE: INFORMED BY FIELD INSPECTOR ON 6-18-20 THAT LOCATION CODE HAS BEEN UPDATED TO "GARR." LOCATION OF "TOXL" DOES NOT EFFECT A BREATH ALCOHOL TEST. LOCATION IS USED FOR INSTRUMENT HISTORY AND DATA COLLECTION FOR D.O.T.**  
*Charles E. Ehen 6-18-20*



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006685  
Location = TOXL      8164.14.00 09/16  
05/07/2020      10:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:01
02 Std. Gas	0.083	10:01
03 Room Air	0.000	10:02
04 Std. Gas	0.083	10:02
05 Room Air	0.000	10:02
06 Std. Gas	0.082	10:03
07 Room Air	0.000	10:03

Lot No = 24119080A1  
Cyl No = 21  
Exp Date = 11/05/2021  
County = 28      Oper No. = 100724



Operator Signature  
CODY MEADOWS

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006685  
Location = TOXL      8164.14.00 09/16  
05/07/2020      10:04

Test	AC	Time
01 Room Air	RFI*	10:05
02 Room Air	RFI*	10:05

\*Invalid Test  
Inhibited - RFI

Sub Name = RFI, TEST Q  
Sub DOB = 10/10/2000  
Sub Sex = Male      Weight = 185  
Test = DUI      Cit = 12345  
Dr. Lic. = ND/123456  
Lot No = 24119080A1  
Cyl No = 21  
Expiration Date = 11/05/2021  
County = 28      Oper No. = 100724

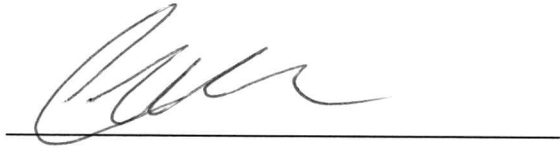

Operator Signature  
CODY MEADOWS

Remarks:

Form 106-I8000