Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer[®] 8000 Serial Number: 80-006682 Location: $To \times L$

- General Setup and Checks: Α.
 - X Diagnostic Tests Pass and Instrument in Ready Mode 1.
 - X Breath tube heated 2.
 - Date, time and location code (Level 2,E). Re-set if necessary. 3.
 - X Print test (Level 1,P). Sign and attach test record. 4.
 - X Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. 5.
 - Display: 365 psi Regulator: 375 psi a.
- Tests (Sign and attach test records): Β.

Configure simulator for the following tests (Level 1,S).

- Wet Calibration Check
 - \square Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.015 AC a. Sim. Ser #: MP 3064 Lot #: 201805C Exp. Date: 5.30.20
 - High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC b. Sim. Ser #: MP3069 Lot #: 20/902 D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

- Interferent Check (Level 1,B): Known Value: 0-/0AC + 0.05% Acetone 2. Sim. Ser #: <u>DR3451</u> Lot #: <u>ICS6</u> Exp. Date: <u>NA</u> Display reads "Interferent Detect"
- RFI Check (CMS Mode) Clisplay reads "RFI Detect" 3.
- Dry Calibration Check (Level 1,C): Known Value 0.080 AC 4. Gas Cyl Lot #:05620000 AI Cyl #: 47 Exp. Date: 4.5.22

 Test 1
 0.080
 Test 4
 0.080
 Test 7
 0.081

 Test 2
 0.081
 Test 5
 0.081
 Test 8
 0.080

 Test 3
 0.081
 Test 6
 0.080
 Test 9
 0.080

Average 0.080

C.

Remarks/Maintenance: <u>ANNUAL INSPECTION</u>

M Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

02/18 Issuing Authority - CEE

<u>5.13.20</u> Date ∧1A

Date

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006682 Location = TOXL 8164.14.00 09/16 05/13/2020 13:06 abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +? Current Instrument Setup Data Entry Mode: Enabled Start Test Sequence: Display Prelim Rslt? DABACABA Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 05620080A1 Standard Cyl #? 47 Standard Expiration? 04/05/2022 Oper No? 666666 Flow Cal. Date: 09/06/2016 Slope 701 -706193 Intercept IR Calibration Date: 09/06/2016 3um 9um 0th Coef(*100):-171501st Coef(*100):2717262nd Coef(*100):831 -23247 134617 1215 H2O adj(mg/l*10k): 393518

Operator Signature CHARLES EDER

Remarks:

RINT TEST

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006682Location = TOXL8164.14.00 09/1605/13/202013:07

	WET CAL CHECK	
Test	AC	Time
01 Room Air	0.000	13:08
02 Std. Sol.	0.012	13:08
03 Room Air	0.000	13:09
04 Std. Sol.	0.012	13:10
05 Room Air	0.000	13:10
06 Std. Sol.	0.013	13:11
07 Room Air	0.000	13:11

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3064 Std Sol No = 201805C County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Low AC 0.015AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006682Location = TOXL8164.14.00 09/1605/13/202013:17

	WET CAL CHECK	1
Test	AC	Time
01 Room Air	0.000	13:18
02 Std. Sol.	0.297	13:18
03 Room Air	0.000	13:19
04 Std. Sol.	0.298	13:20
05 Room Air	0.000	13:20
06 Std. Sol.	0.298	13:21
07 Room Air	0.000	13:21

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3069 Std Sol No = 201902D County = 08 Oper No. = 666666

rn Ú

Operator Signature CHARLES EDER

Remarks:

0.300 AC Form 106-18000

4614 1

Form 106-1800

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006682 Location = TOXL 8164.14.00 09/16 05/13/2020 13:22 Test AC Time 01 Room Air 0.000 13:23 02 *Subject Test INT* 13:24 03 Room Air 0.000 13:24 *Invalid Test Interferent Detected Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = INTERFERENT Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 47Expiration Date = 04/05/2022County = 08Oper No. = 666666

Operator Signature CHARLES EDER

0.05% ACETONE erp

0.10 AC + Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501 CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 SN 80-006682 Location = TOXL 8164.14.00 09/16 05/13/2020 13:25 Test AC Time 01 Diagnostic 13:25 OK 02 Room Air 0.000 13:26 03 *Subject Test RFI* 13:26 04 Room Air 0.000 13:27 *Invalid Test Inhibited - RFI Sub Name = DISCOVER, THE SPIRIT Sub DOB = 02/01/1992Sub Sex = Female Weight = 150Test = DUI Cit = RFI CHECK Dr. Lic. = ND/DIS921456Lot No = 05620080A1Cyl No = 47Expiration Date = 04/05/2022County = 08Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

VECK

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000Alcohol Analyzer
SN 80-006682Location = TOXL
05/13/20208164.14.00 09/16
13:27

			DRY CAL CHECK	
Te	st		AC	Time
01	Room	Air	0.000	13:27
02	Std.	Gas	0.080	13:28
03	Room	Air	0.000	13:28
04	Std.	Gas	0.081	13:29
05	Room	Air	0.000	13:29
06	Std.	Gas	0.081	13:29
07	Room	Air	0.000	13:30

Lot No = 05620080A1 Cyl No = 47 Exp Date = 04/05/2022 County = 08

Oper No. = 666666

1 ankle

Operator Signature CHARLES EDER

ALIBRATION CHECK

0.080AC Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. IntoxilyzerAlcohol AnalyzerNorth Dakota Model 8000SN 80-006682Location = TOXL8164.14.00 09/1605/13/202013:30

	DRY CAL CHECK]
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.080 0.000 0.081 0.000 0.080 0.000	13:31 13:31 13:32 13:32 13:33 13:33 13:33

Lot No = 05620080A1 Cyl No = 47 Exp Date = 04/05/2022 County = 08

Oper No. = 666666

in

Operator Signature CHARLES EDER

CHARLES EDER CALIBRATION CHECK 0.080AC

Form 106-18000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000Alcohol Analyzer
SN 80-006682Location = TOXL8164.14.00 09/16
13:34

	DRY CAL CHECK	
Test	AC	Time
01 Room Air 02 Std. Gas 03 Room Air 04 Std. Gas 05 Room Air 06 Std. Gas 07 Room Air	0.000 0.081 0.000 0.080 0.000 0.080 0.000	13:34 13:35 13:35 13:35 13:36 13:36 13:36 13:37

Lot No = 05620080A1 Cyl No = 47 Exp Date = 04/05/2022 County = 08

Oper No. = 666666

2

Operator Signature CHARLES EDER

BRAMON CHECK

0.080 AC

Form 106-18000