

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006683

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 970 psi Regulator: 975 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: DR5790 Lot #: 20070 Exp. Date: 2-13-22
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2-20-21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS 6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 05620080A1 Cyl #: 47 Exp. Date: 4-5-22

Test 1 <u>0.081</u>	Test 4 <u>0.081</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.081</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.081</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed
Breath Analyst Signature

5/27/20
Date

NA

NA

Reviewed by _____

Date _____

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:35

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:36
02 Std. Sol.	0.017	10:36
03 Room Air	0.000	10:37
04 Std. Sol.	0.017	10:38
05 Room Air	0.000	10:38
06 Std. Sol.	0.018	10:39
07 Room Air	0.000	10:39

08 Sim Temp = 34.0°C

Simul Ser No = DR5190
Std Sol No = 20070
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *Low AC*
 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:40
02 Std. Sol.	0.295	10:41
03 Room Air	0.000	10:42
04 Std. Sol.	0.296	10:43
05 Room Air	0.000	10:43
06 Std. Sol.	0.296	10:44
07 Room Air	0.000	10:44

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:46

Test	AC	Time
01 Room Air	0.000	10:46
02 *Subject Test	INT*	10:47
03 Room Air	0.000	10:47

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 47
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

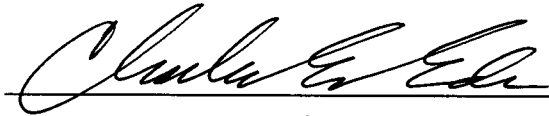

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE
Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:48

Test	AC	Time
01 Diagnostic	OK	10:48
02 Room Air	0.000	10:49
03 *Subject Test	RFI*	10:49
04 Room Air	0.000	10:50

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 47
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:50
02 Std. Gas	0.081	10:51
03 Room Air	0.000	10:51
04 Std. Gas	0.081	10:52
05 Room Air	0.000	10:52
06 Std. Gas	0.081	10:52
07 Room Air	0.000	10:53

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:54
02 Std. Gas	0.081	10:54
03 Room Air	0.000	10:55
04 Std. Gas	0.081	10:55
05 Room Air	0.000	10:55
06 Std. Gas	0.081	10:56
07 Room Air	0.000	10:56

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006683
Location = TOXL 8164.14.00 09/16
05/27/2020 10:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Gas	0.081	10:57
03 Room Air	0.000	10:58
04 Std. Gas	0.081	10:58
05 Room Air	0.000	10:59
06 Std. Gas	0.081	10:59
07 Room Air	0.000	11:00

Lot No = 05620080A1
Cyl No = 47
Exp Date = 04/05/2022
County = 08 Oper No. = 666666

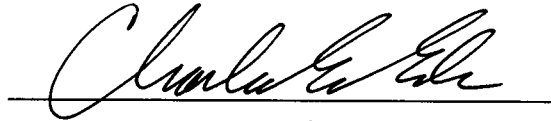

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000