

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006673

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 346 psi Regulator: 350 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3061 Lot #: 201810D Exp. Date: 10.24.20
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2.20.21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 05620080A1 Cyl #: 33 Exp. Date: 4.5.22
Test 1 0.081 Test 4 0.082 Test 7 0.082
Test 2 0.081 Test 5 0.082 Test 8 0.082
Test 3 0.081 Test 6 0.081 Test 9 0.081
Average 0.081

C. Remarks/Maintenance: ANNUAL INSPECTION

NEW EXTERNAL BATTERY

Instrument is acceptable to be used in the field.

Breath Analyst Signature

Reviewed by

5.29.20

Date

Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 09:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:52
02 Std. Sol.	0.019	09:53
03 Room Air	0.000	09:54
04 Std. Sol.	0.019	09:54
05 Room Air	0.000	09:55
06 Std. Sol.	0.019	09:56
07 Room Air	0.000	09:56

08 Sim Temp = 34.0°C

Simul Ser No = MP3061
Std Sol No = 201810D
County = 08 Oper No. = 666666

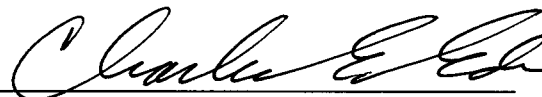

Operator Signature
CHARLES EDER

Remarks:

LOW AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:02

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:02
02 Std. Sol.	0.297	10:03
03 Room Air	0.000	10:04
04 Std. Sol.	0.299	10:04
05 Room Air	0.000	10:05
06 Std. Sol.	0.299	10:06
07 Room Air	0.000	10:06

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:14

Test	AC	Time
01 Room Air	0.000	10:15
02 *Subject Test	INT*	10:16
03 Room Air	0.000	10:16

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 33
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

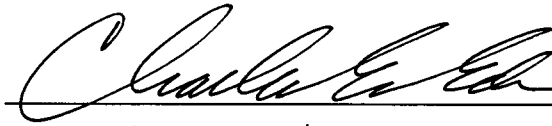

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10 AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:17

Test	AC	Time
01 Diagnostic	OK	10:17
02 Room Air	0.000	10:18
03 *Subject Test	RFI*	10:18
04 Room Air	0.000	10:19

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 33
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

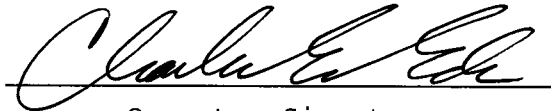

Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:19
02 Std. Gas	0.081	10:20
03 Room Air	0.000	10:20
04 Std. Gas	0.081	10:21
05 Room Air	0.000	10:21
06 Std. Gas	0.081	10:22
07 Room Air	0.000	10:22

Lot No = 05620080A1
Cyl No = 33
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:22

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:23
02 Std. Gas	0.082	10:23
03 Room Air	0.000	10:24
04 Std. Gas	0.082	10:24
05 Room Air	0.000	10:24
06 Std. Gas	0.081	10:25
07 Room Air	0.000	10:25

Lot No = 05620080A1
Cyl No = 33
Exp Date = 04/05/2022
County = 08 Oper No. = 666666

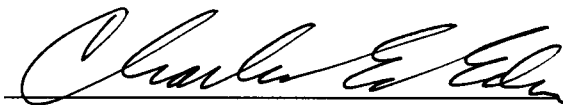

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006673
Location = TOXL 8164.14.00 09/16
05/29/2020 10:26

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:26
02 Std. Gas	0.082	10:27
03 Room Air	0.000	10:27
04 Std. Gas	0.082	10:28
05 Room Air	0.000	10:28
06 Std. Gas	0.081	10:28
07 Room Air	0.000	10:29

Lot No = 05620080A1
Cyl No = 33
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000