

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006672

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 824 psi Regulator: 800 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: MP3061 Lot #: 201810D Exp. Date: 10-24-20
- b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP3069 Lot #: 201902D Exp. Date: 2-20-21

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3451 Lot #: ICS6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect" CEE
4. Dry Calibration Check (Level 1,C): Known Value 4.5.22 AC 0.080 AC
Gas Cyl Lot #: 05620 080 A1 Cyl #: 34 Exp. Date: 4.5.22

Test 1 <u>0.082</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.082</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.082</u>	Test 6 <u>0.082</u>	Test 9 <u>0.081</u>
Average <u>0.082</u>		

C. Remarks/Maintenance: ANNUAL INSPECTION.

REPLACED BREATH HOSE DUE TO BROKEN MOUTHPIECE
STUCK IN END. NEW EXTERNAL BATTERY.

Instrument is acceptable to be used in the field.

Charles E. Ehl
Breath Analyst Signature

6.4.20
Date

NA
Reviewed by

NA
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:25

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05620080A1
Standard Cyl #? 34
Standard Expiration? 04/05/2022
Oper No? 666666

Flow Cal. Date: 07/05/2017
Slope 705
Intercept -687158

IR Calibration Date: 07/05/2017
 3um 9um

0th Coef(*100): -22113 -17955
1st Coef(*100): 271833 130091
2nd Coef(*100): 2129 1363
H2O adj(mg/l*10k): 691 568

***** Printer Test End *****

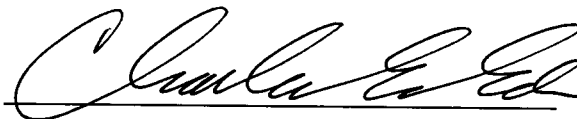

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:27

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:28
02 Std. Sol.	0.019	10:29
03 Room Air	0.000	10:29
04 Std. Sol.	0.019	10:30
05 Room Air	0.000	10:31
06 Std. Sol.	0.020	10:31
07 Room Air	0.000	10:32


08 Sim Temp = 34.0°C

Simul Ser No = MP3061

Std Sol No = 201810D

County = 08

Oper No. = 666666

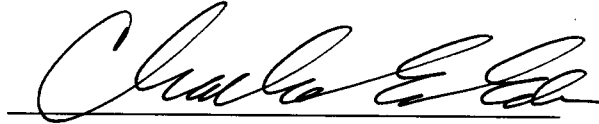

Operator Signature

CHARLES EDER

Remarks:

Low AC
0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:38
02 Std. Sol.	0.296	10:39
03 Room Air	0.000	10:39
04 Std. Sol.	0.298	10:40
05 Room Air	0.000	10:40
06 Std. Sol.	0.298	10:41
07 Room Air	0.000	10:42

08 Sim Temp = 34.0°C

Simul Ser No = MP3069

Std Sol No = 201902D

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:48

Test	AC	Time
01 Room Air	0.000	10:48
02 *Subject Test	INT*	10:49
03 Room Air	0.000	10:49

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 34
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

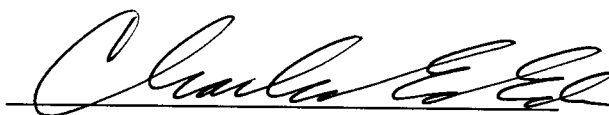

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:50

Test	AC	Time
01 Diagnostic	OK	10:51
02 Room Air	0.000	10:51
03 *Subject Test	RFI*	10:51
04 Room Air	0.000	10:52

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 05620080A1
Cyl No = 34
Expiration Date = 04/05/2022
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:52
02 Std. Gas	0.082	10:53
03 Room Air	0.000	10:53
04 Std. Gas	0.082	10:54
05 Room Air	0.000	10:54
06 Std. Gas	0.082	10:55
07 Room Air	0.000	10:55

Lot No = 05620080A1
Cyl No = 34
Exp Date = 04/05/2022
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:55

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:56
02 Std. Gas	0.082	10:56
03 Room Air	0.000	10:57
04 Std. Gas	0.082	10:57
05 Room Air	0.000	10:58
06 Std. Gas	0.082	10:58
07 Room Air	0.000	10:59

Lot No = 05620080A1

Cyl No = 34

Exp Date = 04/05/2022

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/04/2020 10:59

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:59
02 Std. Gas	0.082	11:00
03 Room Air	0.000	11:00
04 Std. Gas	0.082	11:01
05 Room Air	0.000	11:01
06 Std. Gas	0.081	11:02
07 Room Air	0.000	11:02

Lot No = 05620080A1
Cyl No = 34
Exp Date = 04/05/2022
County = 08 Oper No. = 666666

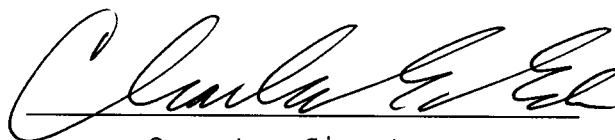

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080AC

Form 106-I8000