

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

| Serial Number 80 - 00 6675 | Instrument Location | |
|---|--|----------------------------------|
| Reason for Install/Repair | 13572 | |
| ☐ Install After Receiving From Crime Laboratory ☐ Inst | all After Location Change | |
| | and the first to | 1. 7 |
| Other (Specify) Amual Certification - Intex re- | rid-19- | (-do) 15/4 ((el on |
| Check When Done: | ,,,, | |
| ☑ 1. Surge Protector Installed/Property Grounded. | | |
| 2. Telephone Line Connected to Intoxilyzer® 8000. | | |
| 3. Breath Tube Heated. | | |
| 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and I | Location; Level 2, Function E). | |
| 5. Scan/Enter Gas Cylinder Information (Level 1, Function | n S). | |
| ☐ 6. Run Tests: | | |
| A. Print Test (Level1, Function P). | | |
| B. ACA Test (Level 1, Function C). | | |
| C. Radio Frequency Interference (RFI) Test (CMS Mo | de or Level 1, Function B or C; Key Radio D | uring Test). |
| 7. Repair and/or Maintenance Performed (if any): | ne. | |
| 8. Complete the Top Portion of the Intoxilyzer® Record (S | FN50496, Form 120-G) and Place it by the | Intoxilyzer® for Use. |
| 9. File Previous Intoxilyzer® Record (SFN504096, Form 1 | 20-G) at the Intoxilyzer® Location at the Ag | ency. |
| 10. Send the Following to the Crime Laboratory: Complete 104-G), Print Test, ACA Test, and RFI Test. | d Intoxilyzer® 8000 Installation and Repair | Checkout (SFN59281, Form |
| Field Inspector Signature | | Date |
| T- 3/10 | Committee that the control of the co | 6-8-20 |
| Crime Laboratory Use Only | | |
| This installation has been reviewed and the instrument is approved | to be used for the analysis of breath to dete | rmine alcohol concentration from |
| he date the Field Inspector performed the installation. This record County of Burleigh, North Dakota, is certified to be a true and correc | | me Laboratory Division, in the |
| | | |
| Reviewed/Certified By | | Certified Date |

FORM 104-G

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = BSPD
06/08/2020
Alcohol Analyzer
SN 80-006675
8164.14.00 09/16

******** Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=| ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup
Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 24119080A1
Standard Cyl #? 31
Standard Expiration? 11/05/2021
Oper No? 069823

Flow Cal. Date: 08/25/2016 Slope 701 Intercept -701324

IR Calibration Date: 08/25/2016
3um 9um

Oth Coef(*100): -8608 -21568
1st Coef(*100): 258710 128604
2nd Coef(*100): 2478 1279
H2O adj(mg/l*10k): 381 528

******* Printer Test End ********

Operator Signature
TIMOTHY BLETH

Remarks:

"frint Test" Install

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = BSPD 06/08/2020

SN 80-006675 8164.14.00 09/16 11:19

DRY CAL CHECK

| Т | est | | AC | Time |
|----|------|-----|-------|-------|
| 01 | Room | Air | 0.000 | 11:20 |
| 02 | Std. | Gas | 0.079 | 11:20 |
| 03 | Room | Air | 0.000 | 11:21 |
| 04 | Std. | Gas | 0.078 | 11:21 |
| 05 | Room | Air | 0.000 | 11:22 |
| 06 | Std. | Gas | 0.079 | 11:22 |
| 07 | Room | Air | 0.000 | 11:23 |

Lot No = 24119080A1

Cyl No = 31

Exp Date = 11/05/2021

County = 08

Oper No. = 069823

Operator Signature TIMOTHY BLETH

Remarks:

'ACATESTI Install

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = BSPD 8164.14.00 09/16 06/08/2020

Alcohol Analyzer SN 80-006675 11:24

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | RFI* | 11:24 |
| 02 Room Air | 0.000 | 11:24 |

*Invalid Test Inhibited - RFI

Lot No = 24119080A1

Cyl No = 31

Exp Date = 11/05/2021


County = 08

Oper No. = 069823

Operator Signature TIMOTHY BLETH

Remarks: RFT Test Install!

Form 106-I8000

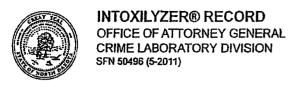

| Intoxilyzer® Serial Number 80 ~ 0066 75 | Location 3 | SPD |
|---|---------------------|--------------------------------|
| Gas Lot Number みくゅう80人 | Gas Cylinder Number | Gas Expiration Date //- 5-202/ |

| | 70,011 | 1 3/ 1 100 |
|-----------|----------------------------------|---|
| Test Date | Chemical Test Operator Number | Operator's Name (PRINT Last Name, First, MI) |
| 6-8-20 | 069823 | Bleth, Timothy J "Print Test" Bleth, Timothy J "ACA" Bleth, Timothy J "ACA" |
| 1 | 069823 | Bleth, Timothy J "Print Test" Bleth, Timothy J "ACA" Bleth, Timothy J "RFI" |
| 6-8-20 | 069823 | Bleth, Timothy 5 "RFI" |
| | | |
| | | |
| | | |
| <u> </u> | | |
| | | · · · · · · · · · · · · · · · · · · · |
| | | · |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | |
| | | · · · · · · · · · · · · · · · · · · · |

| N 80-006675 | Page 5 of 5 | Uploaded 6/12/2020_CEI |
|-------------|-------------|------------------------|
| | | |