OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-00 668 4 Location: 1/0×L
 A. General Setup and Checks: 1. Diagnostic Tests Pass and Instrument in Ready Mode 2. Date tube heated 3. Date, time and location code (Level 2,E). Re-set if necessary. 4. Print test (Level 1,P). Sign and attach test record. 5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi. a. Display: 999 psi Regulator: 1000 psi
 B. Tests (Sign and attach test records):
 Configure dry gas standard for the following tests (Level 1,S). Interferent Check (Level 1,B): Known Value: O·/OAC + 0.05% Acetone Sim. Ser #: DR3451 Lot #: LCS 6 Exp. Date: NA Display reads "Interferent Detect" Interferent Check (Level 1,C): Known Value: NA Display reads "RFI Detect" Interferent Detect Interferent Check (Level 1,B): Known Value: O·OBO AC Interferent Detect Interferent Detect Interferent Detect Interferent Check (Level 1,B): Known Value: O·OBO AC Interferent Detect Inte
C. Remarks/Maintenance: ANNUAL INSPECTION REPLACED BREATH HOSE O-RING
Instrument is acceptable to be used in the field. Breath Analyst Signature Date Reviewed by Date

02/18

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = TOXL 8164.14.00 09/16
06/16/2020 11:06

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-= |
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: Display Prelim Rslt? DABACABA Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? Standard Expiration? 11/05/2021 Oper No? 666666

Flow Cal. Date: 09/06/2016

Slope 709 Intercept -725441

IR Calibration Date: 09/06/2016
3um 9um

Oth Coef(*100): -16013 -23467 1st Coef(*100): 256431 135731 2nd Coef(*100): 2904 1268

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks: RINT TE

H20 adj(mq/l*10k): 553

CMI, Inc. Intoxilyzer North Dakota Model 8000 Location = TOXL 06/16/2020 Alcohol Analyzer SN 80-006684 8164.14.00 09/16 11:09

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:09
02 Std. Sol.	0.017	11:10
03 Room Air	0.000	11:11
04 Std. Sol.	0.018	11:11
05 Room Air	0.000	11:12
06 Std. Sol.	0.018	11:13
07 Room Air	0.000	11:13

08 Sim Temp = 34.0°C

Simul Ser No = MP3064 Std Sol No = 20070

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

LOW AC

Remarks:

Form 106-I8000

0.020 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/16/2020

Alcohol Analyzer SN 80-006684 8164.14.00 09/16 11:14

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:15
02 Std. Sol.	0.250	11:15
03 Room Air	0.000	11:16
04 Std. Sol.	0.251	11:17
05 Room Air	0.000	11:17
06 Std. Sol.	0.251	11:18
07 Room Air	0.000	11:19

08 Sim Temp = 34.0°C

Simul Ser No = MP3067 Std Sol No = 201911B

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

Remarks:

Form 106-I8000

0.250

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006684
Location = TOXL 8164.14.00 09/16
06/16/2020 11:20

Test	AC	Time	
01 Room Air	0.000	11:21	
02 *Subject Test	INT*	11:21	
03 Room Air	0.000	11:22	

*Invalid Test

Interferent Detected

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 9

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks: Interferent Check
0.10AC + 0.05% ACETONE

CMI, Inc. Intoxilyzer	Alcohol Analyzer
North Dakota Model 8000	SN 80-006684
Location = TOXL	8164.14.00 09/16
06/16/2020	11:23

Test	AC	Time	
01 Diagnostic	OK	11:24	
02 Room Air	0.000	11:24	
03 *Subject Test	RFI*	11:24	
04 Room Air	0.000	11:25	

*Invalid Test Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT

Sub DOB = 02/01/1992

Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/DIS921456

Lot No = 24119080A1

Cyl No = 9

Expiration Date = 11/05/2021

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = TOXL 06/16/2020

SN 80-006684 8164.14.00 09/16 11:25

DRY CAL CHECK

Test			AC	Time	
	01	Room	Air	0.000	11:25
	02	Std.	Gas	0.081	11:26
	03	Room	Air	0.000	11:26
	04	Std.	Gas	0.081	11:27
	05	Room	Air	0.000	11:27
	06	Std.	Gas	0.081	11:28
	07	Room	Air	0.000	11:28

Lot No = 24119080A1

Cyl No = 9

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER ALIBRATION CHECK

Remarks:

Form 106-I8000

0.080 AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/16/2020

Alcohol Analyzer SN 80-006684 8164.14.00 09/16 11:28

DRY CAL CHECK

Test			AC		Time	
	01	Room	Air	0.	.000	11:29
	02	Std.	Gas	0.	.081	11:29
	03	Room	Air	0.	.000	11:30
	04	Std.	Gas	0.	.081	11:30
	05	Room	Air	0.	.000	11:31
	06	Std.	Gas	0.	.080	11:31
	07	Room	Air	0.	.000	11:31

Lot No = 24119080A1

Cyl No = 9

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks: CALIBRATION CAECK

0.080AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
06/16/2020

Alcohol Analyzer SN 80-006684 8164.14.00 09/16 11:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:32
02 Std. Gas	0.081	11:32
03 Room Air	0.000	11:33
04 Std. Gas	0.081	11:33
05 Room Air	0.000	11:34
06 Std. Gas	0.081	11:34
07 Room Air	0.000	11:35

Lot No = 24119080A1

Cyl No = 9

Exp Date = 11/05/2021

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Form 106-I8000

0.080 AC