



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) FRANK T. EBERLE		
Location BALMAN County Sheriff's OFC		Intoxilyzer® Serial Number 80-006513
Gas Lot Number 05620080A1	Gas Cylinder Number 008	Gas Expiration Date 4/5/2022

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 A. 0. **082** AC
 B. 0. **081** AC
 C. 0. **083** AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Frank T. Eberle</i>	Date 11/17/2020
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Eberle</i>	Date 11.17.2020


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006513
Location = BOWM 8164.13.00 06/09
11/17/2020 12:25

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:26
02 Std. Gas	0.082	12:26
03 Room Air	0.000	12:26
04 Std. Gas	0.081	12:27
05 Room Air	0.000	12:27
06 Std. Gas	0.083	12:28
07 Room Air	0.000	12:28

Lot No = 05620080A1
Cyl No = 8
Exp Date = 04/05/2022
County = 06 Oper No. = 130015

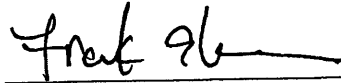

Operator Signature
FRANK EBERLE

Remarks:

Form 106-I8000