



ETHANOL GAS STANDARD CYLINDER REPORT
NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
SFN 59282 (08/2018)

Chemical Test Operator Name (Print) <i>Shalise McBain</i>		
Location <i>Mindt AFB</i>	Intoxilyzer® Serial Number <i>80-006690</i>	
Gas Lot Number <i>24119080A1</i>	Gas Cylinder Number <i>35</i>	Gas Expiration Date <i>11/05/2021</i>

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. *0.081* AC
 - B. *0.081* AC
 - C. *0.081* AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Shalise McBain</i>	Date <i>05 Feb, 2021</i>
Reviewed By (Crime Laboratory Use Only) <i>Charles E. Edwards</i>	Date <i>16 FEB 2021</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006690
Location = MNAF 8164.14.00 09/16
02/05/2021 18:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:39
02 Std. Gas	0.081	18:39
03 Room Air	0.000	18:39
04 Std. Gas	0.081	18:40
05 Room Air	0.000	18:40
06 Std. Gas	0.081	18:41
07 Room Air	0.000	18:41

Lot No = 24119080A1
Cyl No. = 35
Exp Date = 11/05/2021
County = 64 Oper No. = 132656

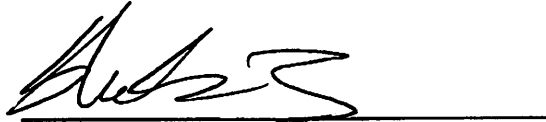

Operator Signature
SHALISE MACBAIN

Remarks:

Form 106-I8000