

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006666

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 349 psi Regulator: 375 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3451 Lot #: FCS 6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

Test 1	<u>0.083</u>	Test 4	<u>0.082</u>	Test 7	<u>0.083</u>
Test 2	<u>0.083</u>	Test 5	<u>0.082</u>	Test 8	<u>0.082</u>
Test 3	<u>0.082</u>	Test 6	<u>0.082</u>	Test 9	<u>0.083</u>
Average <u>0.082</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

4.13.2021  
Date

Kai L. Diet  
Reviewed by

4-13-2021  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006666  
Location = TOXL                    8164.14.00 09/16  
04/13/2021                            11:43

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890- = |  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            24119080A1  
Standard Cyl #?            6  
Standard Expiration?       11/05/2021  
Oper No?                    666666

Flow Cal. Date:            06/19/2019  
Slope                        629  
Intercept                   -588722

IR Calibration Date:        06/19/2019  
                                    3um                    9um

-----  
0th Coef(\*100):            -13823                    -22769  
1st Coef(\*100):            262880                    138031  
2nd Coef(\*100):            3182                      1229  
H2O adj(mg/l\*10k):        247                        351

\*\*\*\*\* Printer Test End \*\*\*\*\*

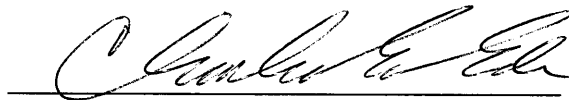

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      11:44

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Sol.	0.019	11:45
03 Room Air	0.000	11:46
04 Std. Sol.	0.020	11:46
05 Room Air	0.000	11:47
06 Std. Sol.	0.020	11:48
07 Room Air	0.000	11:48

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *Low AC*  
                    *0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      11:50

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:50
02 Std. Sol.	0.308	11:51
03 Room Air	0.000	11:52
04 Std. Sol.	0.308	11:53
05 Room Air	0.000	11:53
06 Std. Sol.	0.308	11:54
07 Room Air	0.000	11:54

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 202012A  
County = 08      Oper No. = 666666

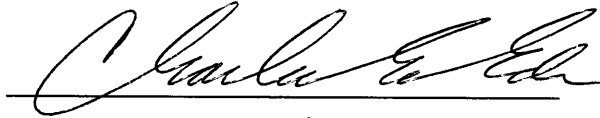

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      11:56

Test	AC	Time
01 Room Air	0.000	11:57
02 *Subject Test	INT*	11:57
03 Room Air	0.000	11:58

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666

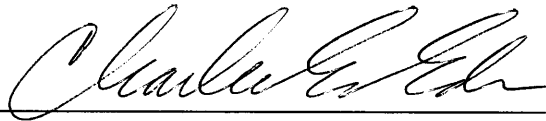

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006666  
Location = TOXL                    8164.14.00 09/16  
04/13/2021                            11:58

Test	AC	Time
01 Diagnostic	OK	11:59
02 Room Air	0.000	11:59
03 *Subject Test	RFI*	12:00
04 Room Air	0.000	12:00

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 02/01/1992  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = RFI CHECK  
Dr. Lic. = ND/DIS921456  
Lot No = 24119080A1  
Cyl No = 6  
Expiration Date = 11/05/2021  
County = 08                            Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

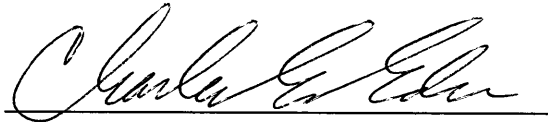

Operator Signature  
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      12:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:01
02 Std. Gas	0.083	12:01
03 Room Air	0.000	12:02
04 Std. Gas	0.083	12:02
05 Room Air	0.000	12:02
06 Std. Gas	0.082	12:03
07 Room Air	0.000	12:03

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      12:05

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:06
02 Std. Gas	0.082	12:06
03 Room Air	0.000	12:06
04 Std. Gas	0.082	12:07
05 Room Air	0.000	12:07
06 Std. Gas	0.082	12:08
07 Room Air	0.000	12:08

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006666  
Location = TOXL      8164.14.00 09/16  
04/13/2021      12:09

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:09
02 Std. Gas	0.083	12:10
03 Room Air	0.000	12:10
04 Std. Gas	0.082	12:10
05 Room Air	0.000	12:11
06 Std. Gas	0.083	12:11
07 Room Air	0.000	12:12

Lot No = 24119080A1  
Cyl No = 6  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000