

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006689 Location: ToxL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 1001 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.015 AC  
Sim. Ser #: MP5319 Lot #: 201911C Exp. Date: 11.19.21
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3.24.23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR5114 Lot #: ICS 6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 24119080A1 Cyl #: 5 Exp. Date: 11.5.21

Test 1	<u>0.078</u>	Test 4	<u>0.078</u>	Test 7	<u>0.078</u>
Test 2	<u>0.078</u>	Test 5	<u>0.078</u>	Test 8	<u>0.078</u>
Test 3	<u>0.078</u>	Test 6	<u>0.079</u>	Test 9	<u>0.079</u>
Average	<u>0.078</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr  
Breath Analyst Signature

4.14.21  
Date

Lali J. Hieb  
Reviewed by

4-19-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006689  
Location = TOXL                    8164.14.00 09/16  
04/14/2021                            12:46

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            05620080A1  
Standard Cyl #?            29  
Standard Expiration?       04/05/2022  
Oper No?                    666666

Flow Cal. Date:            05/29/2020  
Slope                      643  
Intercept                  -576193

IR Calibration Date:       05/29/2020

	3um	9um
0th Coef(*100):	-25870	-28277
1st Coef(*100):	287704	139899
2nd Coef(*100):	26	744
H2O adj(mg/l*10k):	293	328

\*\*\*\*\* Printer Test End \*\*\*\*\*

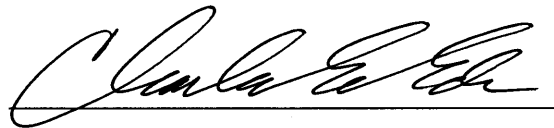

Operator Signature  
CHARLES EDER

Remarks: *PRINT TEST*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006689  
Location = TOXL      8164.14.00 09/16  
04/14/2021      12:53

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:53
02 Std. Sol.	0.014	12:54
03 Room Air	0.000	12:55
04 Std. Sol.	0.013	12:55
05 Room Air	0.000	12:56
06 Std. Sol.	0.014	12:57
07 Room Air	0.000	12:57

08 Sim Temp = 34.0°C

Simul Ser No = MP5319  
Std Sol No = 201911C  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.015 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006689  
Location = TOXL      8164.14.00 09/16  
04/14/2021      12:58

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:59
02 Std. Sol.	0.397	13:00
03 Room Air	0.000	13:00
04 Std. Sol.	0.400	13:01
05 Room Air	0.000	13:02
06 Std. Sol.	0.399	13:02
07 Room Air	0.000	13:03

08 Sim Temp = 34.0°C

Simul Ser No = MP3062  
Std Sol No = 202103E  
County = 08      Oper No. = 666666

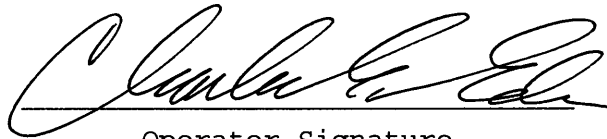

Operator Signature  
CHARLES EDER

Remarks: *HIGH AC*  
*0.400 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006689  
Location = TOXL                    8164.14.00 09/16  
04/14/2021                            13:06

Test	AC	Time
01 Room Air	0.000	13:07
02 *Subject Test	INT*	13:07
03 Room Air	0.000	13:08

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male                            Weight = 150  
Test = DUI                                Cit = INTERFERENT  
Dr. Lic. = ND/DIS871456  
Lot No = 24119080A1  
Cyl No = 5  
Expiration Date = 11/05/2021  
County = 08                                Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006689  
Location = TOXL                    8164.14.00 09/16  
04/14/2021                            13:09

Test	AC	Time
01 Diagnostic	OK	13:09
02 Room Air	0.000	13:10
03 *Subject Test	RFI*	13:10
04 Room Air	0.000	13:11

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male                            Weight = 150  
Test = DUI                                    Cit = RFI CHECK  
Dr. Lic. = ND/DIS871456  
Lot No = 24119080A1  
Cyl No = 5  
Expiration Date = 11/05/2021  
County = 08                                    Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

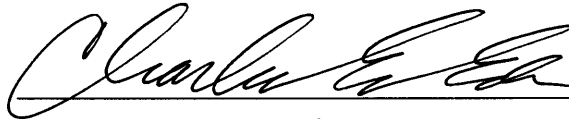

Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006689  
Location = TOXL      8164.14.00 09/16  
04/14/2021      13:12

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:12
02 Std. Gas	0.078	13:12
03 Room Air	0.000	13:13
04 Std. Gas	0.078	13:13
05 Room Air	0.000	13:14
06 Std. Gas	0.078	13:14
07 Room Air	0.000	13:15

Lot No = 24119080A1  
Cyl No = 5  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

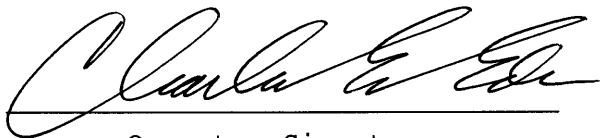

Operator Signature  
CHARLES EDER

Remarks: *CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006689  
Location = TOXL      8164.14.00 09/16  
04/14/2021      13:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:16
02 Std. Gas	0.078	13:16
03 Room Air	0.000	13:16
04 Std. Gas	0.078	13:17
05 Room Air	0.000	13:17
06 Std. Gas	0.079	13:18
07 Room Air	0.000	13:18

Lot No = 24119080A1  
Cyl No = 5  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006689  
Location = TOXL      8164.14.00 09/16  
04/14/2021      13:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:19
02 Std. Gas	0.078	13:20
03 Room Air	0.000	13:20
04 Std. Gas	0.078	13:20
05 Room Air	0.000	13:21
06 Std. Gas	0.079	13:21
07 Room Air	0.000	13:22

Lot No = 24119080A1  
Cyl No = 5  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

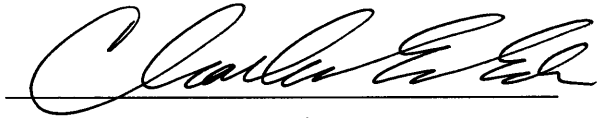

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000