

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006680

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 1000 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2.13.22
 - b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: FC56 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 24119080A1 Cyl #: 6 Exp. Date: 11.5.21

| | | | | | |
|---------|--------------|--------|--------------|--------|--------------|
| Test 1 | <u>0.083</u> | Test 4 | <u>0.083</u> | Test 7 | <u>0.082</u> |
| Test 2 | <u>0.083</u> | Test 5 | <u>0.083</u> | Test 8 | <u>0.083</u> |
| Test 3 | <u>0.082</u> | Test 6 | <u>0.083</u> | Test 9 | <u>0.083</u> |
| Average | <u>0.083</u> | | | | |

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr
Breath Analyst Signature

4.15.21
Date

Kalix J. Niet
Reviewed by

4-19-21
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 11:51

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05620080A1
Standard Cyl #? 32
Standard Expiration? 04/05/2022
Oper No? 666666

Flow Cal. Date: 07/20/2020
Slope 659
Intercept -627239

IR Calibration Date: 07/20/2020
 3um 9um

0th Coef(*100): -19714 -24116
1st Coef(*100): 276675 136189
2nd Coef(*100): 822 883
H2O adj(mg/l*10k): 491 529

***** Printer Test End *****



Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 11:53

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 11:53 |
| 02 Std. Sol. | 0.019 | 11:54 |
| 03 Room Air | 0.000 | 11:55 |
| 04 Std. Sol. | 0.019 | 11:55 |
| 05 Room Air | 0.000 | 11:56 |
| 06 Std. Sol. | 0.020 | 11:57 |
| 07 Room Air | 0.000 | 11:57 |

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 20070
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks: *LOW AC*
 0.020 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:00

WET CAL CHECK

| Test | AC | Time |
|--------------|-------|-------|
| 01 Room Air | 0.000 | 12:01 |
| 02 Std. Sol. | 0.301 | 12:02 |
| 03 Room Air | 0.000 | 12:02 |
| 04 Std. Sol. | 0.304 | 12:03 |
| 05 Room Air | 0.000 | 12:04 |
| 06 Std. Sol. | 0.302 | 12:04 |
| 07 Room Air | 0.000 | 12:05 |


08 Sim Temp = 34.0°C

Simul Ser No = MP5317

Std Sol No = 202012A

County = 08

Oper No. = 666666

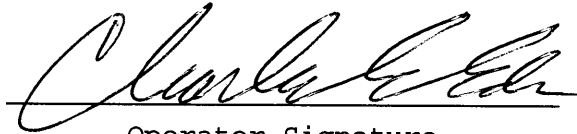

Operator Signature

CHARLES EDER

Remarks:

HIGH AC
0.300 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:08

| Test | AC | Time |
|------------------|-------|-------|
| 01 Room Air | 0.000 | 12:09 |
| 02 *Subject Test | INT* | 12:09 |
| 03 Room Air | 0.000 | 12:10 |

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

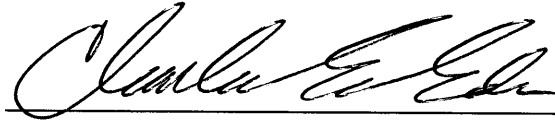

Operator Signature
CHARLES EDER

Remarks:

Interferent Check
0.10AC + 0.05% ACETONE

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:11

| Test | AC | Time |
|------------------|-------|-------|
| 01 Diagnostic | OK | 12:11 |
| 02 Room Air | 0.000 | 12:12 |
| 03 *Subject Test | RFI* | 12:12 |
| 04 Room Air | 0.000 | 12:13 |

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/DIS921456
Lot No = 24119080A1
Cyl No = 6
Expiration Date = 11/05/2021
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:13

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:13 |
| 02 Std. Gas | 0.083 | 12:14 |
| 03 Room Air | 0.000 | 12:14 |
| 04 Std. Gas | 0.083 | 12:15 |
| 05 Room Air | 0.000 | 12:15 |
| 06 Std. Gas | 0.082 | 12:16 |
| 07 Room Air | 0.000 | 12:16 |

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

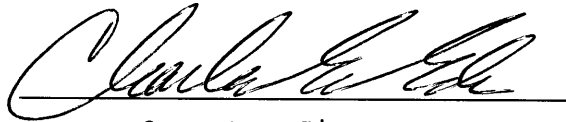

Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:17

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:17 |
| 02 Std. Gas | 0.083 | 12:18 |
| 03 Room Air | 0.000 | 12:18 |
| 04 Std. Gas | 0.083 | 12:18 |
| 05 Room Air | 0.000 | 12:19 |
| 06 Std. Gas | 0.083 | 12:19 |
| 07 Room Air | 0.000 | 12:20 |

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

CALIBRATION CHECK
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006680
Location = TOXL 8164.14.00 09/16
04/15/2021 12:20

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 12:21 |
| 02 Std. Gas | 0.082 | 12:21 |
| 03 Room Air | 0.000 | 12:22 |
| 04 Std. Gas | 0.083 | 12:22 |
| 05 Room Air | 0.000 | 12:22 |
| 06 Std. Gas | 0.083 | 12:23 |
| 07 Room Air | 0.000 | 12:23 |

Lot No = 24119080A1
Cyl No = 6
Exp Date = 11/05/2021
County = 08 Oper No. = 666666

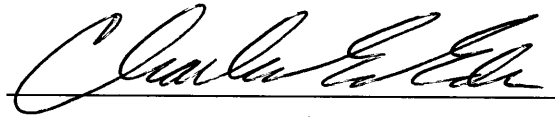

Operator Signature
CHARLES EDER

Remarks: *CALIBRATION CHECKS*
0.080 AC

Form 106-I8000