

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-007087

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 748 psi Regulator: 750 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.030 AC  
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11.19.21
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3.4.23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: ICS 6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 02621080A1 Cyl #: 25 Exp. Date: 3.5.23

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.081</u>
Test 2	<u>0.080</u>	Test 5	<u>0.081</u>	Test 8	<u>0.081</u>
Test 3	<u>0.080</u>	Test 6	<u>0.080</u>	Test 9	<u>0.081</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED EXTERNAL BATTERY

Instrument is acceptable to be used in the field.

Charles E. Edr  
Breath Analyst Signature

4.22.21  
Date

Kari L. Nibb  
Reviewed by

4.27.21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007087  
Location = TOXL                    8164.16.00 09/18  
04/22/2021                            13:04

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-=\_|  
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#\$\$%^&\*()\_+?

Current Instrument Setup


Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?           1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            02621080A1  
Standard Cyl #?            25  
Standard Expiration?       03/05/2023  
Oper No?                    666666

Flow Cal. Date:            07/02/2019  
Slope                        721  
Intercept                   -792109

IR Calibration Date:        07/02/2019  
                                  3um                            9um

-----  
0th Coef(\*100):            -45706                        -18865  
1st Coef(\*100):            253208                        131287  
2nd Coef(\*100):            2874                            1457  
H2O adj(mg/l\*10k):        1171                            483

\*\*\*\*\* Printer Test End \*\*\*\*\*

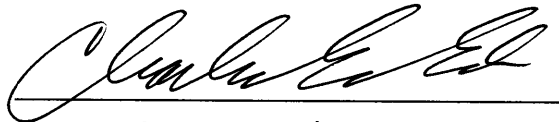

Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:24
02 Std. Sol.	0.026	13:24
03 Room Air	0.000	13:25
04 Std. Sol.	0.027	13:26
05 Room Air	0.000	13:26
06 Std. Sol.	0.027	13:27
07 Room Air	0.000	13:27

08 Sim Temp = 34.0°C

Simul Ser No = MP5289  
Std Sol No = 201911E  
County = 08      Oper No. = 666666

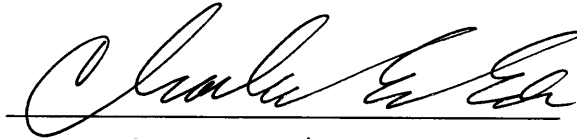

Operator Signature  
CHARLES EDER

Remarks:

Low AC  
~~0.08~~ 0.030 AC  
CEE

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:34

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:34
02 Std. Sol.	0.399	13:35
03 Room Air	0.000	13:36
04 Std. Sol.	0.399	13:37
05 Room Air	0.000	13:37
06 Std. Sol.	0.399	13:38
07 Room Air	0.000	13:38

08 Sim Temp = 34.0°C

Simul Ser No = MP3062

Std Sol No = 202103E

County = 08

Oper No. = 666666



Operator Signature

CHARLES EDER

Remarks:

HIGH AC

0.400 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:40

Test	AC	Time
01 Room Air	0.000	13:40
02 *Subject Test	INT*	13:41
03 Room Air	0.000	13:41

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male      Weight = 150  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/DIS871456  
Lot No = 02621080A1  
Cyl No = 25  
Expiration Date = 03/05/2023  
County = 08      Oper No. = 666666

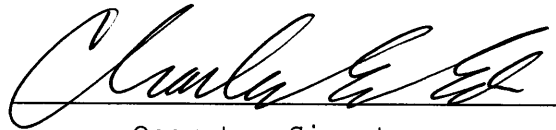

Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007087  
Location = TOXL                    8164.16.00 09/18  
04/22/2021                            13:42

Test	AC	Time
01 Diagnostic	OK	13:43
02 Room Air	0.000	13:43
03 *Subject Test	RFI*	13:43
04 Room Air	0.000	13:44

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/08/1987  
Sub Sex = Male                            Weight = 150  
Test = DUI                                Cit = RFI CHECK  
Dr. Lic. = ND/DIS871456  
Lot No = 02621080A1  
Cyl No = 25  
Expiration Date = 03/05/2023  
County = 08                                Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:45
02 Std. Gas	0.081	13:45
03 Room Air	0.000	13:45
04 Std. Gas	0.080	13:46
05 Room Air	0.000	13:46
06 Std. Gas	0.080	13:47
07 Room Air	0.000	13:47

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

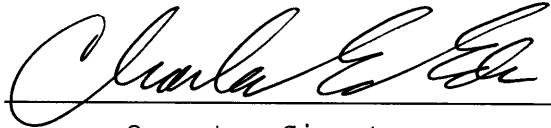

Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*  
*0.080 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:48
02 Std. Gas	0.081	13:48
03 Room Air	0.000	13:49
04 Std. Gas	0.081	13:49
05 Room Air	0.000	13:50
06 Std. Gas	0.080	13:50
07 Room Air	0.000	13:50

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*CALIBRATION CHECK*

*0.080 AC*

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007087  
Location = TOXL      8164.16.00 09/18  
04/22/2021      13:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:51
02 Std. Gas	0.081	13:52
03 Room Air	0.000	13:52
04 Std. Gas	0.081	13:53
05 Room Air	0.000	13:53
06 Std. Gas	0.081	13:53
07 Room Air	0.000	13:54

Lot No = 02621080A1  
Cyl No = 25  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

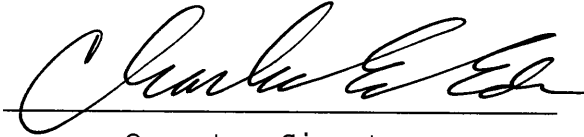

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000