

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-004513

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 726 psi Regulator: 700 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.015 AC  
Sim. Ser #: MP5319 Lot #: 201911C Exp. Date: 11-19-21
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.400 AC  
Sim. Ser #: MP3060 Lot #: 1910B Exp. Date: 10-20-21

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR7348 Lot #: 1155 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 0262108041 Cyl #: 26 Exp. Date: 3-5-23

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.083</u>	Test 5 <u>0.082</u>	Test 8 <u>0.082</u>
Test 3 <u>0.083</u>	Test 6 <u>0.082</u>	Test 9 <u>0.082</u>
Average <u>0.082</u>		

C. Remarks/Maintenance: Annual Inspection.

Left on mountain time for inspection.

Instrument is acceptable to be used in the field.

Kari Y. Aho  
Breath Analyst Signature

5-11-21  
Date

Reviewed by

Charles E. Ed

5-17-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = TOXL                8164.13.00 06/09  
05/11/2021                      12:27

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

Current Instrument Setup

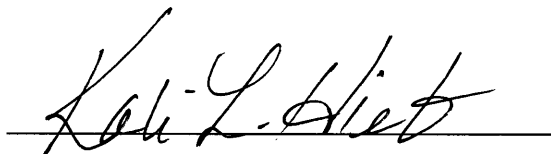
Data Entry Mode:            Enabled  
Start Test Sequence:        DABACABA  
Display Prelim Rslt?        Yes  
Display Third Digit?        Yes  
Inhib Printer(Y/N)?        No  
Display Volume?            No  
Disable On Memfull?        Yes  
# of Print Copies?          1  
Select Std (D/W/I)?        Dry  
Standard Value?            0.080  
Standard Lot #?            05620080A1  
Standard Cyl #?            8  
Standard Expiration?       04/05/2022  
Oper No?                    132598

Flow Cal. Date:            08/20/2015  
Slope                       680  
Intercept                  -696037

IR Calibration Date:       08/21/2015

	3um	9um
0th Coef(*100):	-14436	-13610
1st Coef(*100):	265490	136147
2nd Coef(*100):	4422	1710
H2O adj(mg/l*10k):	672	448

\*\*\*\*\* Printer Test End \*\*\*\*\*

  
Operator Signature  
KALI HIEB

Remarks: *print test*

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

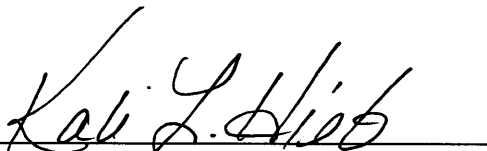
CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006513  
Location = TOXL      8164.13.00 06/09  
05/11/2021      12:31

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:31
02 Std. Sol.	0.015	12:32
03 Room Air	0.000	12:33
04 Std. Sol.	0.016	12:34
05 Room Air	0.000	12:34
06 Std. Sol.	0.015	12:35
07 Room Air	0.000	12:35

08 Sim Temp = 34.0°C

Simul Ser No = MP5319  
Std Sol No = 201911C  
County = 08      Oper No. = 132598



Operator Signature  
KALI HIEB

Remarks:

*low cal check 0.015AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

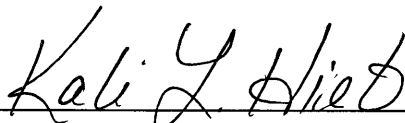
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = TOXL                    8164.13.00 06/09  
05/11/2021                            12:37

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:37
02 Std. Sol.	0.412	12:38
03 Room Air	0.000	12:39
04 Std. Sol.	0.415	12:39
05 Room Air	0.000	12:40
06 Std. Sol.	0.416	12:41
07 Room Air	0.000	12:41

08 Sim Temp = 33.9°C

Simul Ser No = MP3060  
Std Sol No = 19108  
County = 08                            Oper No. = 132598



Operator Signature  
KALI HIEB

Remarks: *High cal check 0.400 AC*

Form 106-I8000

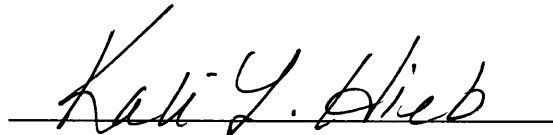
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = TOXL                    8164.13.00 06/09  
05/11/2021                            12:44

Test	AC	Time
01 Room Air	0.000	12:45
02 *Subject Test	INT*	12:46
03 Room Air	0.000	12:47

\*Invalid Test  
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08                            Oper No. = 132598



Operator Signature  
KALI HIEB

Remarks: *Interferent Check*  
*0.10AC + 0.09% Acetone*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

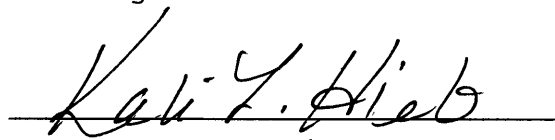
CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = TOXL                8164.13.00 06/09  
05/11/2021                        12:47

Test	AC	Time
01 Diagnostic	OK	12:48
02 Room Air	0.000	12:48
03 Reference	RFI*	12:48
04 Room Air	0.000	12:49

\*Invalid Test  
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT  
Sub DOB = 01/01/1982  
Sub Sex = Female                    Weight = 150  
Test = DUI                            Cit = NA  
Dr. Lic. = ND/DIS821456  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08                          Oper No. = 132598

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

  
Operator Signature  
KALI HIEB

Remarks: *RFI Check*

Form 106-I8000

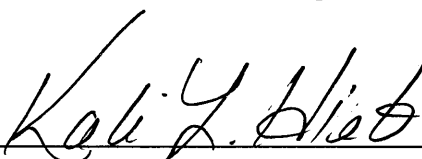
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006513  
Location = TOXL      8164.13.00 06/09  
05/11/2021      12:50

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:50
02 Std. Gas	0.081	12:51
03 Room Air	0.000	12:51
04 Std. Gas	0.083	12:51
05 Room Air	0.000	12:52
06 Std. Gas	0.083	12:52
07 Room Air	0.000	12:53

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 132598

  
\_\_\_\_\_  
Operator Signature  
KALI HIEB

Remarks: *Cal Check 0.080 AC*

Form 106-I8000

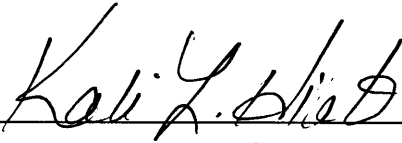
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006513  
Location = TOXL                    8164.13.00 06/09  
05/11/2021                            12:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:54
02 Std. Gas	0.082	12:54
03 Room Air	0.000	12:55
04 Std. Gas	0.082	12:55
05 Room Air	0.000	12:55
06 Std. Gas	0.082	12:56
07 Room Air	0.000	12:56

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08                            Oper No. = 132598

  
\_\_\_\_\_  
Operator Signature  
KALI HIEB

Remarks: *cal check 0.000 AC*

Form 106-I8000



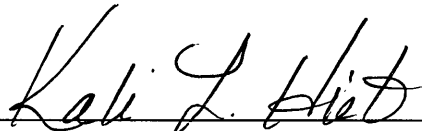
Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006513  
Location = TOXL      8164.13.00 06/09  
05/11/2021      12:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:57
02 Std. Gas	0.082	12:57
03 Room Air	0.000	12:58
04 Std. Gas	0.082	12:58
05 Room Air	0.000	12:59
06 Std. Gas	0.082	12:59
07 Room Air	0.000	13:00

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 132598

  
\_\_\_\_\_  
Operator Signature  
KALI HIEB

Remarks: *Cal check 0.000 AC*

Form 106-I8000