

**INTOXILYZER® 8000 INSPECTION**

Intoxilyzer® 8000 Serial Number: 80-006691

Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 889 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a.  Low AC (Level 1,C): Known Value  $\leq 0.03$  AC: 0.020 AC  
Sim. Ser #: MP3064 Lot #: 20070 Exp. Date: 2-CEE 2.13.22
- b.  High AC (Level 1,C): Known Value  $\geq 0.25$  AC: 0.300 AC  
Sim. Ser #: MP5317 Lot #: 202012A Exp. Date: 12.8.22

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: FCS6 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 2419080A Cyl #: 31 Exp. Date: 11.5.21

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.081</u>	Test 5	<u>0.081</u>	Test 8	<u>0.082</u>
Test 3	<u>0.081</u>	Test 6	<u>0.081</u>	Test 9	<u>0.081</u>
Average	<u>0.081</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ehr  
Breath Analyst Signature

5/21/2021  
Date

Kari L. Lieb  
Reviewed by

5/24/21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-006691  
Location = TOXL                    8164.14.00 09/16  
05/21/2021                            12:16

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

abcdefghijklmnopqrstuvwxy1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&\*()\_+?

Current Instrument Setup  
Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    02621080A1  
Standard Cyl #?                    030  
Standard Expiration?                03/05/2023  
Oper No?                            666666

Flow Cal. Date:                    08/28/2018  
Slope                                699  
Intercept                            -660610

IR Calibration Date:                08/28/2018  
    3um                                9um  
-----  
0th Coef(\*100):                    -9426                                -20764  
1st Coef(\*100):                    258193                                134855  
2nd Coef(\*100):                    3532                                  1465  
H2O adj(mg/l\*10k):                408                                    547

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:                    PRINT TEST

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:17

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:18
02 Std. Sol.	0.017	12:18
03 Room Air	0.000	12:19
04 Std. Sol.	0.018	12:20
05 Room Air	0.000	12:20
06 Std. Sol.	0.018	12:21
07 Room Air	0.000	12:21

08 Sim Temp = 34.0°C

Simul Ser No = MP3064  
Std Sol No = 20070  
County = 08      Oper No. = 666666

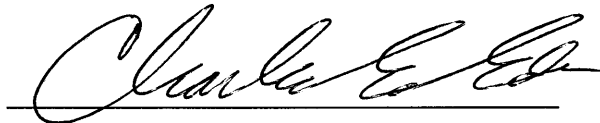

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.020 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:23

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:24
02 Std. Sol.	0.299	12:25
03 Room Air	0.000	12:25
04 Std. Sol.	0.299	12:26
05 Room Air	0.000	12:26
06 Std. Sol.	0.300	12:27
07 Room Air	0.000	12:28

08 Sim Temp = 34.0°C

Simul Ser No = MP5317  
Std Sol No = 202012A  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:      *HIGH AC*  
                    *0.300 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:30

Test	AC	Time
01 Room Air	0.000	12:31
02 *Subject Test	INT*	12:31
03 Room Air	0.000	12:32

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male      Weight = NA  
Test = DUI      Cit = INTERFERENT  
Dr. Lic. = ND/TES989643  
Lot No = 24119080A1  
Cyl No = 31  
Expiration Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks:

*Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-006691  
Location = TOXL                 8164.14.00 09/16  
05/21/2021                        12:32

Test	AC	Time
01 Diagnostic	OK	12:33
02 Room Air	0.000	12:33
03 *Subject Test	RFI*	12:33
04 Room Air	0.000	12:34

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                     Weight = NA  
Test = DUI                         Cit = RFI CHECK  
Dr. Lic. = ND/TES989643  
Lot No = 24119080A1  
Cyl No = 31  
Expiration Date = 11/05/2021  
County = 08                        Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

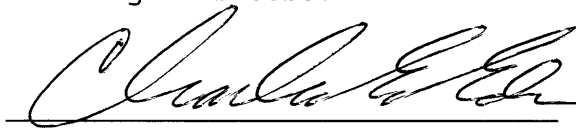

Operator Signature  
CHARLES EDER

Remarks:

*RFI CHECK*

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:34

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:34
02 Std. Gas	0.081	12:35
03 Room Air	0.000	12:35
04 Std. Gas	0.081	12:36
05 Room Air	0.000	12:36
06 Std. Gas	0.081	12:37
07 Room Air	0.000	12:37

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666

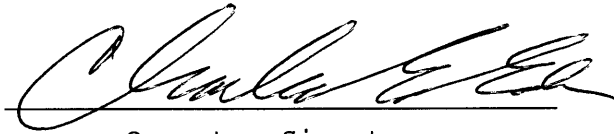

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:38

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:39
02 Std. Gas	0.081	12:39
03 Room Air	0.000	12:40
04 Std. Gas	0.081	12:40
05 Room Air	0.000	12:40
06 Std. Gas	0.081	12:41
07 Room Air	0.000	12:41

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000



Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006691  
Location = TOXL      8164.14.00 09/16  
05/21/2021      12:42

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:43
02 Std. Gas	0.080	12:43
03 Room Air	0.000	12:44
04 Std. Gas	0.082	12:44
05 Room Air	0.000	12:44
06 Std. Gas	0.081	12:45
07 Room Air	0.000	12:45

Lot No = 24119080A1  
Cyl No = 31  
Exp Date = 11/05/2021  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000