Toxicology Section/Breath Alcohol Program Intoxilyzer® 8000 Inspection

INTOXILYZER® 8000 INSPECTION

Intox	cilyzer® 8000 Serial Number: 80-006677 Location: 70×
Α.	General Setup and Checks: 1.
В.	Tests (Sign and attach test records): Configure simulator for the following tests (Level 1,S). Wet Calibration Check a.
C.	Remarks/Maintenance: ANNUAL INSPECTION
	strument is acceptable to be used in the field. 5/26/2/ th Analyst Signature 0-26-71
Revi	ewed by Date

02/18

Issuing Authority - CEE

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006677
Location = TOXL 8164.14.00 09/16
05/26/2021 10:13

******* Printer Test ********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*() +?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled Start Test Sequence: DABACABA Display Prelim Rslt? Yes Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume? No Disable On Memfull? Yes # of Print Copies? 1 Select Std (D/W/I)? Dry Standard Value? 0.080 Standard Lot #? 24119080A1 Standard Cyl #? Standard Expiration? 11/05/2021 Oper No? 666666

Flow Cal. Date: 08/24/2016 Slope 696

IR Calibration Date: 08/24/2016

Oth Coef(*100): -12424 -22862 1st Coef(*100): 263434 131353 2nd Coef(*100): 5070 1477 H2O adj(mg/l*10k): 569 590

3um

-655873

9um

****** Printer Test End *******

Operator Signature CHARLES EDER

Remarks:

Intercept

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/26/2021

Alcohol Analyzer SN 80-006677 8164.14.00 09/16 11:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:21
02 Std. Sol.	0.018	11:22
03 Room Air	0.000	11:22
04 Std. Sol.	0.018	11:23
05 Room Air	0.000	11:23
06 Std. Sol.	0.018	11:24
07 Room Air	0.000	11:25

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP3064 Std Sol No = 20070

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

LOW AC

0.020AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/26/2021

SN 80-006677 8164.14.00 09/16 11:32

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:33
02 Std. Sol.	0.301	11:34
03 Room Air	0.000	11:34
04 Std. Sol.	0.300	11:35
05 Room Air	0.000	11:35
06 Std. Sol.	0.298	11:36
07 Room Air	0.000	11:37

 $08 \text{ Sim Temp} = 34.0^{\circ}\text{C}$

Simul Ser No = MP5317 Std Sol No = 202012A

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006677
Location = TOXL 8164.14.00 09/16
05/26/2021 11:38

Test	AC	Time
01 Room Air	0.000	11:39
02 *Subject Test	INT*	11:39
03 Room Air	0.000	11:40

*Invalid Test

Interferent Detected

Sub Name = TEST, DONOR2 NONE

Sub DOB = 07/25/1998

Sub Sex = Male Weight = NA Test = DUI Cit = INTERFERENT

Dr. Lic. = ND/TES989643

Lot No = 02621080A1

Cyl No = 26

Expiration Date = 03/05/2023

County = 08 Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

Interferent Check 0.10 AC + 0.05% ACETONE FORM 106-18000

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006677
Location = TOXL 8164.14.00 09/16
05/26/2021 11:41

Test	AC	Time
01 Diagnostic	OK	11:42
02 Room Air	0.000	11:43
03 *Subject Test	RFI*	11:43
04 Room Air	0.000	11:44

*Invalid Test Inhibited - RFI

Sub Name = TEST, DONOR2 NONE

Sub DOB = 07/25/1998

Sub Sex = Male Weight = NA Test = DUI Cit = RFI CHECK

Dr. Lic. = ND/TES989643

Lot No = 02621080A1

Cyl No = 26

Expiration Date = 03/05/2023

County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Operator Signature CHARLES EDER

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/26/2021

SN 80-006677 8164.14.00 09/16 11:44

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:44
02 Std. Gas	0.081	11:45
03 Room Air	0.000	11:45
04 Std. Gas	0.081	11:46
05 Room Air	0.000	11:46
06 Std. Gas	0.081	11:47
07 Room Air	0.000	11:47

Lot No = 02621080A1

Cyl No = 26

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature
CHARLES EDER

AUBRATION CHECK

0.080AC

Remarks:

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/26/2021

SN 80-006677 8164.14.00 09/16 11:47

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:48
02 Std. Gas	0.081	11:48
03 Room Air	0.000	11:49
04 Std. Gas	0.082	11:49
05 Room Air	0.000	11:50
06 Std. Gas	0.081	11:50
07 Room Air	0.000	11:50

Lot No = 02621080A1

Cyl No = 26

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature CHARLES EDER

Remarks:

CALIBRATION CHECK

0.080AC

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = TOXL
05/26/2021

SN 80-006677 8164.14.00 09/16 11:51

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:51
02 Std. Gas	0.081	11:52
03 Room Air	0.000	11:52
04 Std. Gas	0.081	11:53
05 Room Air	0.000	11:53
06 Std. Gas	0.081	11:53
07 Room Air	0.000	11:54

Lot No = 02621080A1

Cyl No = 26

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

Operator Signature

CHARLES EDER

Remarks: