

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-007086 Location: TOXL

A. General Setup and Checks:

1.  Diagnostic Tests Pass and Instrument in Ready Mode
2.  Breath tube heated
3.  Date, time and location code (Level 2,E). Re-set if necessary.
4.  Print test (Level 1,P). Sign and attach test record.
5.  Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
  - a. Display: 971 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check
  - a.  Low AC (Level 1,C): Known Value  $\leq$  0.03 AC: 0.030 AC  
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11.19.21
  - b.  High AC (Level 1,C): Known Value  $\geq$  0.25 AC: 0.400 AC  
Sim. Ser #: MP3062 Lot #: 202103E Exp. Date: 3.24.23

Configure dry gas standard for the following tests (Level 1,S).

2.  Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone  
Sim. Ser #: DR3847 Lot #: IC56 Exp. Date: NA  
 Display reads "Interferent Detect"
3.  RFI Check (CMS Mode)  Display reads "RFI Detect"
4.  Dry Calibration Check (Level 1,C): Known Value 0.080 AC  
Gas Cyl Lot #: 0262108041 Cyl #: 26 Exp. Date: 3.5.23

Test 1	<u>0.081</u>	Test 4	<u>0.082</u>	Test 7	<u>0.082</u>
Test 2	<u>0.081</u>	Test 5	<u>0.082</u>	Test 8	<u>0.081</u>
Test 3	<u>0.081</u>	Test 6	<u>0.082</u>	Test 9	<u>0.081</u>
Average <u>0.081</u>					

C. Remarks/Maintenance: ANNUAL INSPECTION

Instrument is acceptable to be used in the field.

Charles E. Ed  
Breath Analyst Signature

6/14/21  
Date

Kari G. Dieb  
Reviewed by

6-14-21  
Date

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007086  
Location = TOXL                    8164.16.00 09/18  
06/14/2021                            11:48

\*\*\*\*\* Printer Test \*\*\*\*\*

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

abcdefghijklmnopqrstuvwxyz1234567890-|=|  
ABCDEFGHIJKLMNopqrstuvwxyz!@#%&^\*()\_+?

Current Instrument Setup

Data Entry Mode:                    Enabled  
Start Test Sequence:                DABACABA  
Display Prelim Rslt?                Yes  
Display Third Digit?                Yes  
Inhib Printer(Y/N)?                No  
Display Volume?                    No  
Disable On Memfull?                Yes  
# of Print Copies?                  1  
Select Std (D/W/I)?                Dry  
Standard Value?                    0.080  
Standard Lot #?                    05620080A1  
Standard Cyl #?                    44  
Standard Expiration?                04/05/2022  
Oper No?                            666666

Flow Cal. Date:                    10/16/2018  
Slope                                688  
Intercept                            -653049

IR Calibration Date:                10/16/2018

	3um	9um
0th Coef(*100):	-47466	-23263
1st Coef(*100):	233222	130299
2nd Coef(*100):	3639	1618
H2O adj(mg/l*10k):	1280	608

\*\*\*\*\* Printer Test End \*\*\*\*\*



Operator Signature  
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.16.00 09/18  
06/14/2021      12:01

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:01
02 Std. Sol.	0.027	12:02
03 Room Air	0.000	12:03
04 Std. Sol.	0.028	12:04
05 Room Air	0.000	12:04
06 Std. Sol.	0.028	12:05
07 Room Air	0.000	12:05

08 Sim Temp = 34.0°C

Simul Ser No = MP5289  
Std Sol No = 201911E  
County = 08      Oper No. = 666666

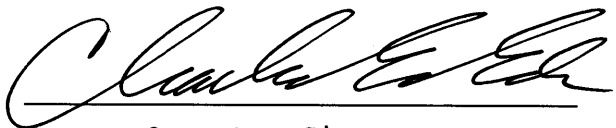

Operator Signature  
CHARLES EDER

Remarks: *Low AC*  
*0.030 AC*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.16.00 09/18  
06/14/2021      12:12

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:13
02 Std. Sol.	0.406	12:14
03 Room Air	0.000	12:15
04 Std. Sol.	0.406	12:15
05 Room Air	0.000	12:16
06 Std. Sol.	0.404	12:17
07 Room Air	0.000	12:17


08 Sim Temp = 34.0°C

Simul Ser No = MP3062

Std Sol No = 202103E

County = 08

Oper No. = 666666

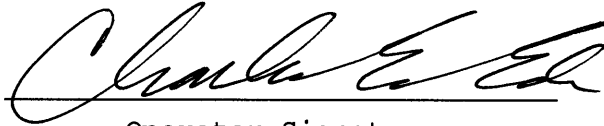

Operator Signature  
CHARLES EDER

Remarks:

HIGH AC  
0.400AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000        SN 80-007086  
Location = TOXL                 8164.16.00 09/18  
06/14/2021                        12:24

Test	AC	Time
01 Room Air	0.000	12:24
02 *Subject Test	INT*	12:25
03 Room Air	0.000	12:25

\*Invalid Test  
Interferent Detected

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                     Weight = NA  
Test = DUI                         Cit = INTERFERENT  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08                        Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: *Interferent Check*  
*0.10 AC + 0.05% ACETONE*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer            Alcohol Analyzer  
North Dakota Model 8000            SN 80-007086  
Location = TOXL                    8164.16.00 09/18  
06/14/2021                            12:26

Test	AC	Time
01 Diagnostic	OK	12:26
02 Room Air	0.000	12:27
03 *Subject Test	RFI*	12:27
04 Room Air	0.000	12:28

\*Invalid Test  
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE  
Sub DOB = 07/25/1998  
Sub Sex = Male                            Weight = NA  
Test = DUI                                    Cit = RFI CHECK  
Dr. Lic. = ND/TES989643  
Lot No = 02621080A1  
Cyl No = 26  
Expiration Date = 03/05/2023  
County = 08                                    Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

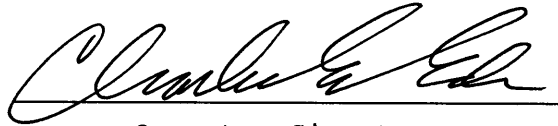

Operator Signature  
CHARLES EDER

Remarks: *RFI CHECK*

Form 106-I8000

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.16.00 09/18  
06/14/2021      12:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:20
02 Std. Gas	0.081	12:20
03 Room Air	0.000	12:21
04 Std. Gas	0.081	12:21
05 Room Air	0.000	12:22
06 Std. Gas	0.081	12:22
07 Room Air	0.000	12:23

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666



Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080 AC

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.16.00 09/18  
06/14/2021      12:28

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:29
02 Std. Gas	0.082	12:29
03 Room Air	0.000	12:30
04 Std. Gas	0.082	12:30
05 Room Air	0.000	12:31
06 Std. Gas	0.082	12:31
07 Room Air	0.000	12:32

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

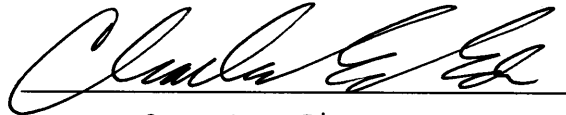

Operator Signature  
CHARLES EDER

Remarks:

CALIBRATION CHECK  
0.080 AC

Form 106-I8000




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-007086  
Location = TOXL      8164.16.00 09/18  
06/14/2021      12:32

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:32
02 Std. Gas	0.082	12:33
03 Room Air	0.000	12:33
04 Std. Gas	0.081	12:34
05 Room Air	0.000	12:34
06 Std. Gas	0.081	12:34
07 Room Air	0.000	12:35

Lot No = 02621080A1  
Cyl No = 26  
Exp Date = 03/05/2023  
County = 08      Oper No. = 666666

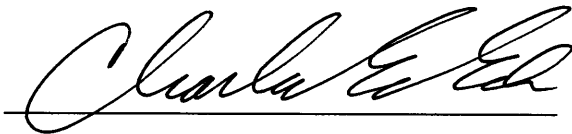

Operator Signature  
CHARLES EDER

Remarks: CALIBRATION CHECK  
0.080AC

Form 106-I8000