

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006682

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 918 psi Regulator: 900 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.030 AC
Sim. Ser #: MP5289 Lot #: 201911E Exp. Date: 11.19.21
 - b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.300 AC
Sim. Ser #: MP5317 Lot #: 202001B Exp. Date: 1.28.22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR3847 Lot #: ICSG6 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080 Al Cyl #: 26 Exp. Date: 3.5.23

Test 1	<u>0.081</u>	Test 4	<u>0.081</u>	Test 7	<u>0.080</u>
Test 2	<u>0.080</u>	Test 5	<u>0.080</u>	Test 8	<u>0.081</u>
Test 3	<u>0.080</u>	Test 6	<u>0.080</u>	Test 9	<u>0.080</u>
Average	<u>0.080</u>				

C. Remarks/Maintenance: ANNUAL INSPECTION, REPLACED EXTERNAL BATTERY, REPLACED GAS REGULATOR ASSEMBLY DUE TO SENSOR NOT TARING TO ZERO.

Instrument is acceptable to be used in the field.

Charles E. Ea
Breath Analyst Signature

6/23/21
Date

Kali L. Hiest
Reviewed by

6.28.21
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 15:12

***** Printer Test *****

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^ &* () _ + ?

abcdefghijklmnopqrstuvwxy1234567890- = |
ABCDEFGHIJKLMN OPQRSTUVWXYZ!@# \$%^ &* () _ + ?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 05620080A1
Standard Cyl #? 45
Standard Expiration? 04/05/2022
Oper No? 666666

Flow Cal. Date: 09/06/2016
Slope 701
Intercept -706193

IR Calibration Date: 09/06/2016

	3um	9um
0th Coef(*100):	-17150	-23247
1st Coef(*100):	271726	134617
2nd Coef(*100):	831	1215
H2O adj(mg/l*10k):	393	518

***** Printer Test End *****

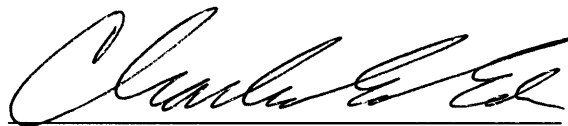

Operator Signature
CHARLES EDER

Remarks:

PRINT TEST

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 15:40

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:41
02 Std. Sol.	0.028	15:42
03 Room Air	0.000	15:42
04 Std. Sol.	0.028	15:43
05 Room Air	0.000	15:43
06 Std. Sol.	0.028	15:44
07 Room Air	0.000	15:45

08 Sim Temp = 34.0°C

Simul Ser No = MP5289
Std Sol No = 201911E
County = 08 Oper No. = 666666



Operator Signature
CHARLES EDER

Remarks:

LOW AC
0.030 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 15:52

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	15:53
02 Std. Sol.	0.294	15:54
03 Room Air	0.000	15:54
04 Std. Sol.	0.295	15:55
05 Room Air	0.000	15:55
06 Std. Sol.	0.293	15:56
07 Room Air	0.000	15:57

08 Sim Temp = 34.0°C

Simul Ser No = MP5317
Std Sol No = 202001B
County = 08 Oper No. = 666666

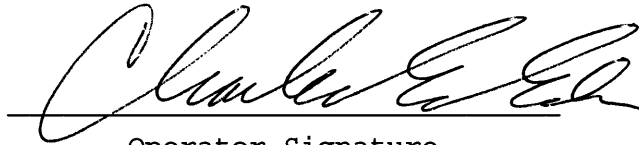

Operator Signature
CHARLES EDER

Remarks: *HIGH AC*
0.300 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 16:09

Test	AC	Time
01 Diagnostic	OK	16:10
02 Room Air	0.000	16:10
03 *Subject Test	RFI*	16:11
04 Room Air	0.000	16:11

*Invalid Test
Inhibited - RFI

Sub Name = TEST, DONOR2 NONE
Sub DOB = 07/25/1998
Sub Sex = Male Weight = NA
Test = DUI Cit = RFI CHECK
Dr. Lic. = ND/TES989643
Lot No = 02621080A1
Cyl No = 26
Expiration Date = 03/05/2023
County = 08 Oper No. = 666666

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

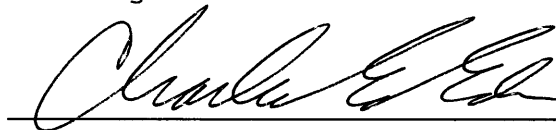

Operator Signature
CHARLES EDER

Remarks:

RFI CHECK

Form 106-I8000


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 16:15

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:16
02 Std. Gas	0.081	16:16
03 Room Air	0.000	16:16
04 Std. Gas	0.080	16:17
05 Room Air	0.000	16:17
06 Std. Gas	0.080	16:18
07 Room Air	0.000	16:18

Lot No = 02621080A1
Cyl No = 26
Exp Date = 03/05/2023
County = 08 Oper No. = 666666

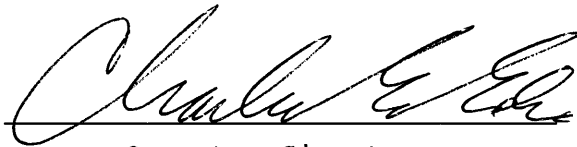

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080 AC

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006682
Location = TOXL 8164.14.00 09/16
06/23/2021 16:19

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	16:19
02 Std. Gas	0.080	16:20
03 Room Air	0.000	16:20
04 Std. Gas	0.081	16:21
05 Room Air	0.000	16:21
06 Std. Gas	0.080	16:21
07 Room Air	0.000	16:22


Lot No = 02621080A1

Cyl No = 26

Exp Date = 03/05/2023

County = 08

Oper No. = 666666

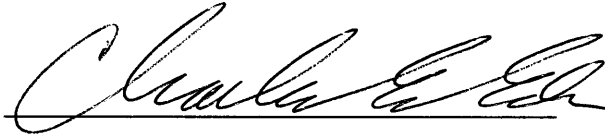

Operator Signature
CHARLES EDER

Remarks: CALIBRATION CHECK
0.080AC

Form 106-I8000