

INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59281 (06/2018)

Serial Number	Instrument Location BCCC	
80 - 00 66 82 Reason for Install/Repair	Dece	
☐ Install After Receiving From Crime Laboratory ☐ Install Afte	er Location Change	
Other (Specify)	-	
Check When Done:		
1. Surge Protector Installed/Property Grounded.		
汉 2. Telephone Line Connected to Intoxilyzer® 8000.		
4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Locati	ion; Level 2, Function E).	
5. Scan/Enter Gas Cylinder Information (Level 1, Function S).		
6. Run Tests:		
A. Print Test (Level1, Function P).		
☑ B. ACA Test (Level 1, Function C).		
	Level 1, Function B or C; Key Radio During Tes	st).
7. Repair and/or Maintenance Performed (if any):		
8. Complete the Top Portion of the Intoxilyzer® Record (SFN56	0496, Form 120-G) and Place it by the Intoxilyze	er® for Use.
☑ 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G)		
10. Send the Following to the Crime Laboratory: Completed Into 104-G), Print Test, ACA Test, and RFI Test.	oxilyzer® 8000 Installation and Repair Checkout	(SFN59281, Form
Field Inspecto Signature	Date	1/15/2021
Crime Laboratory Use Only		
This installation has been reviewed and the instrument is approved to be the date the Field Inspector performed the installation. This record on fill County of Burleigh, North Dakota, is certified to be a true and correct co	le at the Office of Attorney General, Crime Labor	

FORM 104-G

Reviewed/Certified By

Certified Date 9 • 15 • 202 J

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = BCCC
09/15/2021

Alcohol Analyzer
SN 80-006682
8164.14.00 09/16
13:28

********* Printer Test *********

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=|
ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Enabled Data Entry Mode: Start Test Sequence: DABACABA Yes Display Prelim Rslt? Display Third Digit? Yes Inhib Printer(Y/N)? No Display Volume?
Disable On Memfull? No Yes # of Print Copies? Select Std (D/W/I)? Dry 0.080 Standard Value? Standard Lot #? 05620080A1 Standard Cyl #? 45 Standard Expiration? 04/05/2022 100012 Oper No?

Flow Cal. Date: 09/06/2016 Slope 701 Intercept -706193

IR Calibration Date: 09/06/2016
3um 9um

Oth Coef(*100): -17150 -23247
1st Coef(*100): 271726 134617
2nd Coef(*100): 831 1215
H2O adj(mg/l*10k): 393 518

******* Printer Test End ********

Operator Signature DANA RUSTEBAKKE

Remarks: Print Test Good

Inter mered from old juil to new joil.

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = BCCC
09/15/2021

Alcohol Analyzer SN 80-006682 8164.14.00 09/16 13:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:30
02 Std. Gas	0.080	13:30
03 Room Air	0.000	13:31
04 Std. Gas	0.080	13:31
05 Room Air	0.000	13:32
06 Std. Gas	0.081	13:32
07 Room Air	0.000	13:32

Lot No = 05620080A1

Cyl No = 45

Exp Date = 04/05/2022

County = 02

Oper No. = 100012

Operator Signature DANA RUSTEBAKKE

Remarks:

ACA Test Good

Infor moved from old jail to new jail.

Form 106-I8000

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer
North Dakota Model 8000
Location = BCCC
09/15/2021

Alcohol Analyzer SN 80-006682 8164.14.00 09/16 13:33

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:33
02 Std. Gas	RFI*	13:34
03 Room Air	0.000	13:34

*Invalid Test Inhibited - RFI

Lot No = 05620080A1

Cyl No = 45

Exp Date = 04/05/2022

County = 02

Oper No. = 100012

Operator Signature DANA RUSTEBAKKE

Zutelle

Remarks:

RFI TEXT GUOD

Index moved from all juil to new juil.

Form 106-I8000