




INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006509	Instrument Location Grand Forks AFB
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): N/A
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature 	Date 25 Jan 22
---	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By 	Certified Date 27 JAN 2022
--	--------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = GFAF 8164.14.00 09/16
01/25/2022 10:35

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=-|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 12
Standard Expiration? 03/05/2023
Oper No? 132924

Flow Cal. Date: 06/11/2019
Slope 445
Intercept 431160

IR Calibration Date: 06/11/2019
 3um 9um

0th Coef(*100): -27266 -14145
1st Coef(*100): 276865 139368
2nd Coef(*100): 2326 1158
H2O adj(mg/l*10k): 584 288

***** Printer Test End *****



Operator Signature
ZACHARY BRODBECK

Remarks:


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = GFAF 8164.14.00 09/16
01/25/2022 10:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:39
02 Std. Gas	0.079	10:39
03 Room Air	0.000	10:40
04 Std. Gas	0.079	10:40
05 Room Air	0.000	10:41
06 Std. Gas	0.078	10:41
07 Room Air	0.000	10:42

Lot No = 02621080A1
Cyl No = 12
Exp Date = 03/05/2023
County = 65 Oper No. = 132924


Operator Signature
ZACHARY BRODBECK

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006509
Location = GFAF 8164.14.00 09/16
01/25/2022 10:45

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	10:46
02 Room Air	RFI*	10:46

*Invalid Test
Inhibited - RFI

Lot No = 02621080A1
Cyl No = 12
Exp Date = 03/05/2023
County = 65 Oper No. = 132924

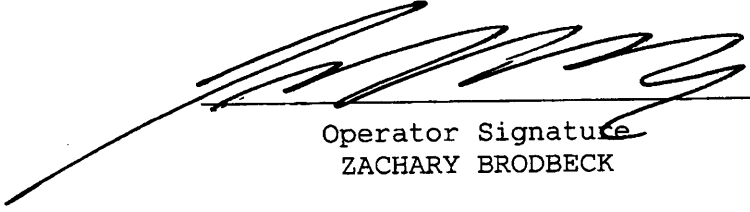

Operator Signature
ZACHARY BRODBECK

Remarks:

Form 106-I8000

