

ETHANOL GAS STANDARD CYLINDER REPORT

NORTH DAKOTA OFFICE OF ATTORNEY GENERAL CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM SFN 59282 (08/2018)

Chemical Test Operator Name (Print) MATTHEW ANDERSOD					
Location RICHLAND COUNTY JAIL		Intoxilyzer® Serial Number 80 - 00 650 8			
Gas Lot Number © 2 621 080 A 1	Gas Cylinder Number 33		Gas Expiration Date		

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0.0 80 AC
 - B. 0.08 0 AC
 - C. 0.**⊘%**⊘ AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- ¥ 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature	Date
1/2 the fluson	2-08-2022
Reviewed By (Crime Laboratory Use Only)	Date 2/1/
(Mallet It	2/14/2022

Intoxilyzer Test Record and Checklist NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer North Dakota Model 8000 Location = WAHP 02/08/2022

SN 80-006508 8164.14.00 09/16 14:48

DRY CAL CHECK

02 Std. Gas	49 49 50 50 51 51

Lot No = 02621080A1

Cyl No = 33

Exp Date = 03/05/2023

County = 39

Oper No. = 020401

Operator Signature MATTHEW ANDERSON

Remarks:

Form 106-I8000