

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006678

Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 980 psi Regulator: 1000 psi

B. Tests (Sign and attach test records):

Configure simulator for the following tests (Level 1,S).

1. Wet Calibration Check

- a. Low AC (Level 1,C): Known Value ≤ 0.03 AC: 0.020 AC
Sim. Ser #: MP3070 Lot #: 22CP41184 Exp. Date: 3/1/2024
- b. High AC (Level 1,C): Known Value ≥ 0.25 AC: 0.358 AC 0.400
Sim. Ser #: MP3065 Lot #: 202108E Exp. Date: 3/24/2023

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5114 Lot #: ICSG Exp. Date: NM
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 02621080A1 Cyl #: 31 Exp. Date: 3/29/3/15/2023

Test 1 <u>0.080</u>	Test 4 <u>0.080</u>	Test 7 <u>0.081</u>
Test 2 <u>0.081</u>	Test 5 <u>0.080</u>	Test 8 <u>0.080</u>
Test 3 <u>0.081</u>	Test 6 <u>0.080</u>	Test 9 <u>0.080</u>
Average <u>0.080</u>		

C. Remarks/Maintenance: Annual Inspection.

Replaced O-ring on breath tube hose and simulator return.

Instrument is acceptable to be used in the field.

Janelli Portschiller
Breath Analyst Signature

5/17/22
Date

[Signature]
Reviewed by

060622
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = TOXL 8164.14.00 09/16
05/17/2022 09:20

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:20
02 Std. Sol.	0.018	09:23
03 Room Air	0.000	09:24
04 Std. Sol.	0.018	09:25
05 Room Air	0.000	09:25
06 Std. Sol.	0.018	09:26
07 Room Air	0.000	09:26

08 Sim Temp = 34.0°C

Simul Ser No = MP3070
Std Sol No = 22CP41184
County = 08 Oper No. = 132128


Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low AC check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = TOXL 8164.14.00 09/16
05/17/2022 09:28

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:29
02 Std. Sol.	0.391	09:30
03 Room Air	0.000	09:30
04 Std. Sol.	0.393	09:31
05 Room Air	0.000	09:31
06 Std. Sol.	0.393	09:32
07 Room Air	0.000	09:33

08 Sim Temp = 34.0°C

Simul Ser No = MP3065

Std Sol No = 202103E

County = 08

Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *High AC Check 0.400*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = TOXL 8164.14.00 09/16
05/17/2022 09:53

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:53
02 Std. Gas	0.080	09:54
03 Room Air	0.000	09:54
04 Std. Gas	0.080	09:55
05 Room Air	0.000	09:55
06 Std. Gas	0.080	09:55
07 Room Air	0.000	09:56

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry calibration check #4-6*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006678
Location = TOXL 8164.14.00 09/16
05/17/2022 09:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	09:57
02 Std. Gas	0.081	09:57
03 Room Air	0.000	09:58
04 Std. Gas	0.080	09:58
05 Room Air	0.000	09:59
06 Std. Gas	0.080	09:59
07 Room Air	0.000	10:00

Lot No = 02621080A1
Cyl No = 31
Exp Date = 03/05/2023
County = 08 Oper No. = 132128

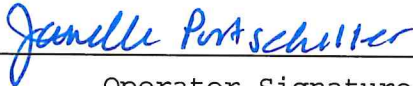

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry calibration checks # 7-9*