



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number <i>80-006688</i>	Instrument Location <i>STEE (Lidder Co So)</i>
Reason for Install/Repair	
<input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): *N/A*
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

*210 July 27
pp*

Field Inspector Signature <i>Barry Vannatta</i>	Date <i>6-16-2022</i>
--	--------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Janelle Porschiller</i>	Certified Date <i>06/21/22</i>
---	-----------------------------------

Note: I called Barry Vannatta on 7/21/22 to verify a form 120-G was placed by the instrument at the time of the instrument installation. He said one was placed by the instrument. yet 21 July 22

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
06/16/2022 18:02

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 34
Standard Expiration? 03/05/2023
Oper No? 039909

Flow Cal. Date: 09/07/2016
Slope 698
Intercept -642832

IR Calibration Date: 09/07/2016
 3um 9um

	3um	9um
0th Coef(*100):	-10822	-23152
1st Coef(*100):	262468	134492
2nd Coef(*100):	3796	1436
H2O adj(mg/l*10k):	665	568

***** Printer Test End *****



Operator Signature
BARRY VANNATTA

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006688
Location = STEE 8164.14.00 09/16
06/16/2022 18:04

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	18:04
02 Std. Gas	0.079	18:05
03 Room Air	0.000	18:05
04 Std. Gas	0.078	18:06
05 Room Air	0.000	18:06
06 Std. Gas	0.079	18:07
07 Room Air	0.000	18:07


Lot No = 02621080A1

Cyl No = 34

Exp Date = 03/05/2023

County = 22

Oper No. = 039909

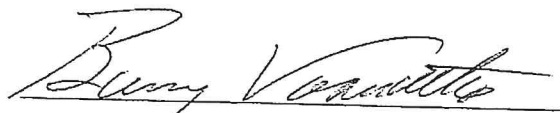

Operator Signature
BARRY VANNATTA

Remarks:

Form 106-I8000

