

INTOXILYZER® 8000 INSPECTION

Intoxilyzer® 8000 Serial Number: 80-006672 Location: TOXL

A. General Setup and Checks:

1. Diagnostic Tests Pass and Instrument in Ready Mode
2. Breath tube heated
3. Date, time and location code (Level 2,E). Re-set if necessary.
4. Print test (Level 1,P). Sign and attach test record.
5. Tank monitor (Level 3,D,G). Display and Regulator within 50 psi.
 - a. Display: 881 psi Regulator: 850 psi

B. Tests (Sign and attach test records):

- Configure simulator for the following tests (Level 1,S).
1. Wet Calibration Check
 - a. Low AC (Level 1,C): Known Value \leq 0.03 AC: 0.020 AC
Sim. Ser #: MP3060 Lot #: 202201A Exp. Date: 1/11/24
 - b. High AC (Level 1,C): Known Value \geq 0.25 AC: 0.300 AC
Sim. Ser #: MP3064 Lot #: 202012A Exp. Date: 12/8/22

Configure dry gas standard for the following tests (Level 1,S).

2. Interferent Check (Level 1,B): Known Value: 0.10 AC + 0.05% Acetone
Sim. Ser #: DR5114 Lot #: 1656 Exp. Date: NA
 Display reads "Interferent Detect"
3. RFI Check (CMS Mode) Display reads "RFI Detect"
4. Dry Calibration Check (Level 1,C): Known Value 0.080 AC
Gas Cyl Lot #: 34921080A1 Cyl #: 49 Exp. Date: 2/5/24

Test 1 <u>0.081</u>	Test 4 <u>0.082</u>	Test 7 <u>0.082</u>
Test 2 <u>0.080</u>	Test 5 <u>0.080</u>	Test 8 <u>0.081</u>
Test 3 <u>0.081</u>	Test 6 <u>0.080</u>	Test 9 <u>0.081</u>
Average <u>0.081</u>		

C. Remarks/Maintenance: Annual inspection.

Replaced breath tube + simulator return o-rings.
Hard to supply sample for interferent check. Per prior inspections
the instrument has a flow sensor issue and it is hard to supply a sample.
This instrument is for classroom use only.

No Instrument is acceptable to be used in the field. NO

Janelle Putschiller
Breath Analyst Signature

06/27/22
Date

Reviewed by _____

062922
Date

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 10:45

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%&^*()_+?

Current Instrument Setup


Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 25
Standard Expiration? 03/05/2023
Oper No? 132128

Flow Cal. Date: 05/18/2021
Slope 589
Intercept 439222

IR Calibration Date: 07/05/2017
 3um 9um

	3um	9um
0th Coef(*100):	-22113	-17955
1st Coef(*100):	271833	130091
2nd Coef(*100):	2129	1363
H2O adj(mg/l*10k):	691	568

***** Printer Test End *****



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Print test*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 10:47

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:48
02 Std. Sol.	0.019	10:48
03 Room Air	0.000	10:49
04 Std. Sol.	0.020	10:50
05 Room Air	0.000	10:50
06 Std. Sol.	0.019	10:51
07 Room Air	0.000	10:51

08 Sim Temp = 34.0°C

Simul Ser No = MP3060
Std Sol No = 202201A
County = 08 Oper No. = 132128

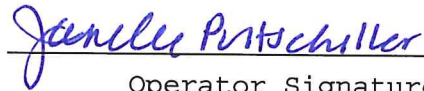

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Low Ac check 0.020*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 10:57

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:57
02 Std. Sol.	0.295	10:58
03 Room Air	0.000	10:58
04 Std. Sol.	0.296	10:59
05 Room Air	0.000	11:00
06 Std. Sol.	0.296	11:00
07 Room Air	0.000	11:01

08 Sim Temp = 34.0°C

Simul Ser No = MP3064
Std Sol No = 202012A
County = 08 Oper No. = 132128

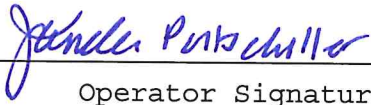
Operator Signature
JANELLE PORTSCHELLER

Remarks: *High Ac check 0.300*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 11:31

Test	AC	Time
01 Room Air	0.000	11:32
02 *Subject Test	INT*	11:33
03 Room Air	0.000	11:34

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = INTERFERENT CK
Dr. Lic. = ND/DIS821456
Lot No = 34921080A1
Cyl No = 49
Expiration Date = 02/05/2024
County = 08 Oper No. = 132128



Operator Signature
JANELLE PORTSCHELLER

Remarks: *Interferent Check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 11:46

Test	AC	Time
01 Diagnostic	OK	11:47
02 Room Air	0.000	11:48
03 *Subject Test	RFI*	11:49
04 Room Air	0.000	11:50

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 01/01/1982
Sub Sex = Female Weight = 150
Test = DUI Cit = RFI CHK
Dr. Lic. = ND/DIS821456
Lot No = 34921080A1
Cyl No = 49
Expiration Date = 02/05/2024
County = 08 Oper No. = 132128

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

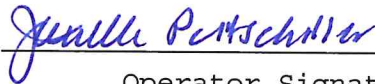

Operator Signature
JANELLE PORTSCHELLER

Remarks: *RFI check*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 11:52

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:53
02 Std. Gas	0.081	11:53
03 Room Air	0.000	11:54
04 Std. Gas	0.080	11:54
05 Room Air	0.000	11:55
06 Std. Gas	0.081	11:55
07 Room Air	0.000	11:56

Lot No = 34921080A1
Cyl No = 49
Exp Date = 02/05/2024
County = 08 Oper No. = 132128

Janelle Portscheller

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 1-3*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 11:56

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:57
02 Std. Gas	0.082	11:57
03 Room Air	0.000	11:57
04 Std. Gas	0.080	11:58
05 Room Air	0.000	11:58
06 Std. Gas	0.080	11:59
07 Room Air	0.000	11:59

Lot No = 34921080A1
Cyl No = 49
Exp Date = 02/05/2024
County = 08 Oper No. = 132128

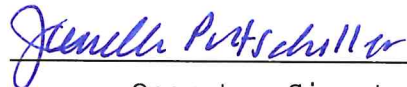

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check #1-3^{Jan} 4-6*


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006672
Location = TOXL 8164.14.00 09/16
06/27/2022 12:01

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	12:01
02 Std. Gas	0.082	12:02
03 Room Air	0.000	12:02
04 Std. Gas	0.081	12:03
05 Room Air	0.000	12:03
06 Std. Gas	0.081	12:04
07 Room Air	0.000	12:04

Lot No = 34921080A1
Cyl No = 49
Exp Date = 02/05/2024
County = 08 Oper No. = 132128

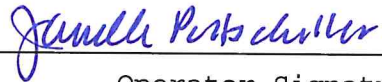

Operator Signature
JANELLE PORTSCHELLER

Remarks: *Dry Calibration Check # 7-9*