



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59281 (06/2018)

Serial Number 80-006687	Instrument Location Powers Lake PD
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date 07-12-2022
-------------------------------------------------	---------------------------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By <i>Janelle Pertschauer</i>	Certified Date 7/13/2022
-----------------------------------------------------	------------------------------------

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = PLPD 8164.14.00 09/16
07/12/2022 04:30

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNPOQRSTUVWXYZ!@#%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 02621080A1
Standard Cyl #? 28
Standard Expiration? 03/05/2023
Oper No? 132720

Flow Cal. Date: 10/05/2018
Slope 696
Intercept -609977

IR Calibration Date: 10/05/2018
3um 9um

	3um	9um
0th Coef(*100):	-11478	-18771
1st Coef(*100):	255256	132135
2nd Coef(*100):	4432	1676
H2O adj(mg/l*10k):	604	575

***** Printer Test End *****



Operator Signature
BRYTON ALLARD

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = PLPD 8164.14.00 09/16
07/12/2022 04:30

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	04:31
02 Std. Gas	0.079	04:31
03 Room Air	0.000	04:32
04 Std. Gas	0.079	04:32
05 Room Air	0.000	04:33
06 Std. Gas	0.079	04:33
07 Room Air	0.000	04:34


Lot No = 02621080A1

Cyl No = 28

Exp Date = 03/05/2023

County = 07

Oper No. = 132720

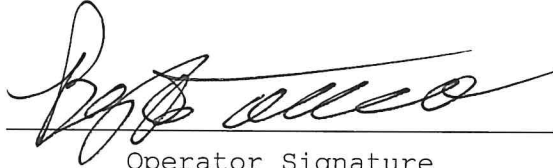

Operator Signature
BRYTON ALLARD

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = PLPD 8164.14.00 09/16
07/12/2022 04:35

Test	AC	Time
01 Room Air	0.000	04:36
02 Subject Test 1	0.000	04:37
03 Room Air	AMB*	04:37
04 Room Air	0.000	04:38

*Invalid Test
Check Ambient Conditions

Sub Name = TEST, RFI NA
Sub DOB = 01/01/2000
Sub Sex = Male Weight = NA
Test = OTH Cit = NA
Dr. Lic. = ND/NA
Lot No = 02621080A1
Cyl No = 28
Expiration Date = 03/05/2023
County = 07 Oper No. = 132720



Operator Signature
BRYTON ALLARD

Remarks:

Form 106-I8000

* Note I called Bryton on 7/13/2022 to ask why an ambient fail was obtained. Bryton said he accidentally had a mouth piece on the instrument. The mouth piece on the instrument caused the ambient fail. JPA 07/13/22

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006687
Location = PLPD 8164.14.00 09/16
07/12/2022 04:39

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	04:39
02 Std. Gas	0.079	04:40
03 Room Air	0.000	04:40
04 Std. Gas	RFI*	04:40
05 Room Air	RFI*	04:41

*Invalid Test
Inhibited - RFI


Lot No = 02621080A1

Cyl No = 28

Exp Date = 03/05/2023

County = 07

Oper No. = 132720



Operator Signature
BRYTON ALLARD

Remarks:

Form 106-I8000