

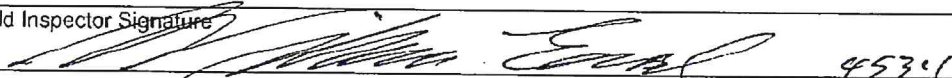


**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM  
 SFN 59281 (06/2018)

Serial Number	80- <del>00671</del> <sup>006671</sup> <sub>12 Sept. 22</sub>	Instrument Location	Devils Lake LEC
Reason for Install/Repair			
<input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____			

Check When Done:

- 1. Surge Protector Installed/Property Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. Radio Frequency Interference (RFI) Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if any): Moved Location
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN504096, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer® 8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature		Date	9/9/2022
---------------------------	--	------	----------

Crime Laboratory Use Only

This installation has been reviewed and the instrument is approved to be used for the analysis of breath to determine alcohol concentration from the date the Field Inspector performed the installation. This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Reviewed/Certified By		Certified Date	9/12/2022
-----------------------	---	----------------	-----------




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006671  
Location = DEVL      8164.14.00 09/16  
09/09/2022      10:08

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	10:08
02 Std. Gas	0.081	10:09
03 Room Air	0.000	10:09
04 Std. Gas	0.081	10:10
05 Room Air	0.000	10:10
06 Std. Gas	0.081	10:10
07 Room Air	0.000	10:11

Lot No = 26021080A1  
Cyl No = 19  
Exp Date = 10/05/2023  
County = 36      Oper No. = 132008

  
Operator Signature 4589  
WILLIAM EARL

Remarks:

Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501


CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-006671  
Location = DEVL      8164.14.00 09/16  
09/09/2022      10:11

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	10:12
02 Room Air	0.000	10:12

\*Invalid Test  
Inhibited - RFI

Lot No = 26021080A1  
Cyl No = 19  
Exp Date = 10/05/2023  
County = 36      Oper No. = 132008

  
Operator Signature      4939  
WILLIAM EARL

Remarks: