



ETHANOL GAS STANDARD CYLINDER REPORT
 NORTH DAKOTA OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION-TOXICOLOGY SECTION/BREATH ALCOHOL PROGRAM
 SFN 59282 (08/2018)

Chemical Test Operator Name (Print) Jeffrey Bauske		
Location DEVL	Intoxilyzer® Serial Number 80-006671	
Gas Lot Number 34921080A1	Gas Cylinder Number 46	Gas Expiration Date 02/05/2024

Check When Done:

- 1. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 2. Perform an ACA Test (Level 1, Function C).
 Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).
 - A. 0. 0.53 AC
 - B. 0. 0.8 AC
 - C. 0. 0.8 AC
- 3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 5. Send the Following to the Crime Laboratory:
 - A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).
 - B. ACA Test Record.

Chemical Test Operator Signature <i>Jeffrey Bauske 287</i>	Date 10/16/2022
Reviewed By (Crime Laboratory Use Only) <i>[Signature]</i>	Date 17OCT2022

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-006671
Location = DEVL 8164.14.00 09/16
10/16/2022 11:06

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	11:07
02 Std. Gas	0.082	11:07
03 Room Air	0.000	11:08
04 Std. Gas	0.082	11:08
05 Room Air	0.000	11:09
06 Std. Gas	0.082	11:09
07 Room Air	0.000	11:09

Lot No = 34921080A1
Cyl No = 46
Exp Date = 02/05/2024
County = 36 Oper No. = 120501



Operator Signature
JEFFREY BAUSKE

Remarks: *NEW CYLINDER ACA TEST*