



**INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT**  
 OFFICE OF ATTORNEY GENERAL  
 CRIME LABORATORY DIVISION  
 Toxicology Section/Breath Alcohol Program  
 SFN 59281 (5-2011)

Serial Number <i>80-005949</i>	Instrument Location <i>GFAF</i>
Reason for Install/Repair <input type="checkbox"/> Install After Receiving From Crime Laboratory <input checked="" type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
  - A. Print Test (Level 1, Function P).
  - B. ACA Test (Level 1, Function C).
  - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): \_\_\_\_\_
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>[Signature]</i>	Date <i>4 Jan 16</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>11 Jan 16</i>

This record on file at the Office of Attorney General, Crime Laboratory Division, in the County of Burleigh, North Dakota, is certified to be a true and correct copy of the documents received.

Certified By: *[Signature]* Certified Date: *18 Nov. 2016*




Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005949  
Location = GFAF      8164.13.00 06/09  
01/04/2016      17:54

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	17:55
02 Std. Gas	0.078	17:56
03 Room Air	0.000	17:56
04 Std. Gas	0.078	17:56
05 Room Air	0.000	17:57
06 Std. Gas	0.079	17:57
07 Room Air	0.000	17:58

Lot No = 19615080A5  
Cyl No = 30  
Exp Date = 09/05/2017  
County = 65      Oper No. = 120011

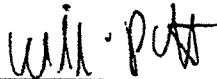

Operator Signature  
WILLIAM PETTY

Remarks:

Form 106-I8000


Intoxilyzer Test Record and Checklist  
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer      Alcohol Analyzer  
North Dakota Model 8000      SN 80-005949  
Location = GFAP      8164.13.00 06/09  
01/04/2016      17:59

Test	AC	Time
01 Room Air	0.000	18:00
02 *Subject Test	RFI*	18:01
03 Room Air	0.000	18:01

\*Invalid Test  
Inhibited - RFI

Sub Name = PETTY, WILLIAM R  
Sub DOB = 10/28/1962  
Sub Sex = Male      Weight = 210  
Test = OTH      Cit = NA  
Dr. Lic. = ND/PET629806  
Lot No = 19615080A5  
Cyl No = 30  
Expiration Date = 09/05/2017  
County = 65      Oper No. = 120011

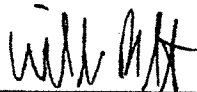

---

Operator Signature  
WILLIAM PETTY

Remarks:

Form 106-I8000