



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (5-2011)

| | |
|-------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------------|------------------------------------------------|
| Serial Number <i>80-004204</i> | Instrument Location <i>Cass County Jail</i> |
| Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____ | |

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (Use CMS Mode; Key Radio During First Room Air).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

| | |
|---------------------------------------------------------------|---------------------------|
| Field Inspector Signature <i>S Getz</i> | Date <i>01/09/2012</i> |
| Reviewed By (Crime Laboratory Use Only) <i>Deb Kashner</i> | Date <i>03 Apr 12</i> |

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CASS 8164.13.00 06/09
01/09/2012 10:38

***** Printer Test *****

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

abcdefghijklmnopqrstuvwxyz1234567890-=_|
ABCDEFGHIJKLMNopqrstuvwxyz!@#\$\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
Start Test Sequence: DABACABA
Display Prelim Rslt? Yes
Display Third Digit? Yes
Inhib Printer(Y/N)? No
Display Volume? No
Disable On Memfull? Yes
of Print Copies? 1
Select Std (D/W/I)? Dry
Standard Value? 0.080
Standard Lot #? 23411080A1
Standard Cyl #? 2
Standard Expiration? 10/01/2013
Oper No? 020535

Flow Cal. Date: 07/13/2010
Slope 750
Intercept -815939

IR Calibration Date: 07/14/2011
 3um 9um

0th Coef(*100): -9686 -4209
1st Coef(*100): 259274 128761
2nd Coef(*100): 3107 1691
H2O adj(mg/l*10k): 532 548

***** Printer Test End *****



Operator Signature
SHAWN GETZ

Remarks: *Print test for install*

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CASS 8164.13.00 06/09
01/09/2012 10:45

DRY CAL CHECK

| Test | AC | Time |
|-------------|-------|-------|
| 01 Room Air | 0.000 | 10:46 |
| 02 Std. Gas | 0.080 | 10:46 |
| 03 Room Air | 0.000 | 10:47 |
| 04 Std. Gas | 0.081 | 10:47 |
| 05 Room Air | 0.000 | 10:48 |
| 06 Std. Gas | 0.080 | 10:48 |
| 07 Room Air | 0.000 | 10:49 |

Lot No = 23411080A1
Cyl No = 2
Exp Date = 10/01/2013
County = 09 Oper No. = 020535



Operator Signature
SHAWN GETZ

Remarks:

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004204
Location = CASS 8164.13.00 06/09
01/09/2012 10:41

| Test | AC | Time |
|---------------|-------|-------|
| 01 Diagnostic | OK | 10:44 |
| 02 Room Air | RFI* | 10:44 |
| 03 Room Air | 0.000 | 10:44 |

*Invalid Test
Inhibited - RFI

Sub Name = RFI, TEST NA
Sub DOB = 01/01/1970
Sub Sex = Female Weight = NA
Test = DUI Cit = NA
Dr. Lic. = ND/NA
Lot No = 23411080A1
Cyl No = 2
Expiration Date = 10/01/2013
County = 09 Oper No. = 020535

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.



Operator Signature
SHAWN GETZ

Remarks: *RFI test*