



INTOXILYZER® 8000 INSTALLATION AND REPAIR CHECKOUT
 OFFICE OF ATTORNEY GENERAL
 CRIME LABORATORY DIVISION
 Toxicology Section/Breath Alcohol Program
 SFN 59281 (03-2014)

Serial Number 80-004186	Instrument Location Oliver Co. SO.
Reason for Install/Repair <input checked="" type="checkbox"/> Install After Receiving From Crime Laboratory <input type="checkbox"/> Install After Location Change <input type="checkbox"/> Other (Specify) _____	

Check When Done:

- 1. Surge Protector Installed/Properly Grounded.
- 2. Telephone Line Connected to Intoxilyzer® 8000.
- 3. Breath Tube Heated.
- 4. Enter Preliminary Data (i.e. Date, Time, DST (Y), and Location; Level 2, Function E).
- 5. Scan/Enter Gas Cylinder Information (Level 1, Function S).
- 6. Run Tests:
 - A. Print Test (Level 1, Function P).
 - B. ACA Test (Level 1, Function C).
 - C. RFI Test (CMS Mode or Level 1, Function B or C; Key Radio During Test).
- 7. Repair and/or Maintenance Performed (if Any): _____
- 8. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.
- 9. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.
- 10. Send the Following to the Crime Laboratory: Completed Intoxilyzer®8000 Installation and Repair Checkout (SFN59281, Form 104-G), Print Test, ACA Test, and RFI Test.

Field Inspector Signature <i>Grant Dilsch</i>	Date 1-8-16
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date 11 Jan 16

Intoxilyzer Test Record and Checklist
 NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
 North Dakota Model 8000 SN 80-004186
 Location = OLIV 8164.13.00 06/09
 01/08/2016 19:35

***** Printer Test *****

abcdefghijklmnopqrstu vwxyz1234567890- = |
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

abcdefghijklmnopqrstu vwxyz1234567890- = |
 ABCDEFGHIJKLMNOPQRSTUVWXYZ!@#\$%^&*()_+?

Current Instrument Setup

Data Entry Mode: Enabled
 Start Test Sequence: DABACABA
 Display Prelim Rslt? Yes
 Display Third Digit? Yes
 Inhib Printer(Y/N)? No
 Display Volume? No
 Disable On Memfull? Yes
 # of Print Copies? 1
 Select Std (D/W/I)? Dry
 Standard Value? 0.080
 Standard Lot #? 19615080A5
 Standard Cyl #? 33
 Standard Expiration? 09/05/2017
 Oper No? 020904

Flow Cal. Date: 04/22/2014
 Slope 589
 Intercept -71396

IR Calibration Date: 12/10/2015

	3um	9um
0th Coef(*100):	-37549	-32658
1st Coef(*100):	272347	137615
2nd Coef(*100):	2049	1261
H2O adj(mg/l*10k):	399	507

***** Printer Test End *****



Operator Signature
 SHANNON P. LIEBOLD

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = OLIV 8164.13.00 06/09
01/08/2016 19:36

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	19:37
02 Std. Gas	0.080	19:37
03 Room Air	0.000	19:38
04 Std. Gas	0.081	19:38
05 Room Air	0.000	19:39
06 Std. Gas	0.080	19:39
07 Room Air	0.000	19:39

Lot No = 19615080A5
Cyl No = 33
Exp Date = 09/05/2017
County = 33 Oper No. = 020904



Operator Signature
SHANNON P. LIEBOLD

Remarks:

Form 106-I8000

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer . Alcohol Analyzer
North Dakota Model 8000 SN 80-004186
Location = OLIV 8164.13.00 06/09
01/08/2016 19:40

DRY CAL CHECK

Test	AC	Time
01 Room Air	RFI*	19:40
02 Room Air	0.000	19:41

*Invalid Test
Inhibited - RFI

Lot No = 19615080A5
Cyl No = 33
Exp Date = 09/05/2017
County = 33 Oper No. = 020904



Operator Signature
SHANNON P. LIEBOLD

Remarks:

Form 106-I8000