

INTOXILYZER 8000
REPAIR AND MAINTENANCE

Serial No.: 80-003057

Location: TOXL

Check When Done

1. Is a three-pronged grounded outlet used? ✓
2. Is the warm-up time less than 20 minutes? ✓
3. Is the breath tube heated? ✓
4. Is the diagnostic check complete? ✓
5. Is the time, date, and year correct (re-set if necessary)? ✓
6. Print test. (Attach test record.) ✓
7. Calibration Check. Use Ethanol Breath Standard (EBS). Do three sets in ACA mode. (Attach test records.)

Lot No. 15410080A1 Cylinder No. 4 Expir. Date 07/01/12

Test 1 0.079 Test 4 0.080 Test 7 0.079

Test 2 0.080 Test 5 0.080 Test 8 0.081

Test 3 0.080 Test 6 0.080 Test 9 0.080

Average 0.080

8. Low AC. Use < 0.03 AC in ACA mode. (Attach test record.) ✓
9. Linearity Test. Use ≥ 0.25 AC in ACA mode. (Attach test record.) ✓
10. Interferent check. Use 0.05% acetone plus 0.10 AC ethanol in ABA mode. (Attach test record.) ✓
11. RFI check. Run CMS mode. Key radio on first room air. (Attach test record.) ✓

80-003057

P. 120 of 129

Electrical Repairs:

1. Was the calibration adjusted/performed? (Attach test record.) @ CM1
2. Was the light source changed? NO
3. Was the sample chamber cleaned? NO

Other Maintenance/Repairs:

1. Was the printer cleaned/repaired? NO
2. Was the solenoid cleaned? NO
3. Was new tubing installed? NO
4. Were additional parts replaced/changed/repaired? YES

5. Remarks/Maintenance record: CM1 "replaced the broken simulator return port assy," and "tightened up the cal inlet & reset the cell gains."

18 Jan 2011
Date

Deb Kashur
Field Inspector's Signature

18 Jan 2011
Date

Maryn Pearson
Reviewed by

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 13:43

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:43
02 Std. Gas	0.079	13:44
03 Room Air	0.000	13:44
04 Std. Gas	0.080	13:44
05 Room Air	0.000	13:45
06 Std. Gas	0.080	13:45
07 Room Air	0.000	13:46

Lot No = 15410080A1
Cyl No = 04
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-18000

80-003057
P. 123 of 129
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 13:49

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	13:50
02 Std. Gas	0.080	13:50
03 Room Air	0.000	13:51
04 Std. Gas	0.080	13:51
05 Room Air	0.000	13:52
06 Std. Gas	0.080	13:52
07 Room Air	0.000	13:53

Lot No = 15410080A1

Cyl No = 04

Exp Date = 07/01/2012

County = 08

Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003057
P 124 of 129
PK

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 14:00

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:01
02 Std. Gas	0.079	14:01
03 Room Air	0.000	14:01
04 Std. Gas	0.081	14:02
05 Room Air	0.000	14:02
06 Std. Gas	0.080	14:03
07 Room Air	0.000	14:03

Lot No = 15410080A1
Cyl No = 04
Exp Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Calibration check 0.080 AC

Form 106-I8000

80-003057
P. 125-129
JFM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 14:41

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:42
02 Std. Sol.	0.020	14:43
03 Room Air	0.000	14:43
04 Std. Sol.	0.020	14:44
05 Room Air	0.000	14:45
06 Std. Sol.	0.020	14:45
07 Room Air	0.000	14:46

08 Sim Temp = 34.0°C

Simul Ser No = 00000

Std Sol No = 000

County = 08

Oper No. = 777777

Deb Kasher

Operator Signature
DEB KASHUR

Remarks:

Low AC

0.021 AC

Form 106-I8000

80-003057
P. 126 of 129
FM

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 14:22

WET CAL CHECK

Test	AC	Time
01 Room Air	0.000	14:23
02 Std. Sol.	0.293	14:24
03 Room Air	0.000	14:24
04 Std. Sol.	0.293	14:25
05 Room Air	0.000	14:26
06 Std. Sol.	0.293	14:26
07 Room Air	0.000	14:27

08 Sim Temp = 34.0°C

Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

Linearity test 0.294 AC

Form 106-18000

80-003057
P.127-1129
YH

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 15:30

Test	AC	Time
01 Room Air	0.000	15:31
02 *Subject Test	INT*	15:31
03 Room Air	0.000	15:32

*Invalid Test
Interferent Detected

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Lot No = 15410080A1
Cyl No = 4
Expiration Date = 07/01/2012
County = 08 Oper No. = 777777

Deb Kashur

Operator Signature
DEB KASHUR

Remarks:

Interferent check 0.10 AC + acetone

Form 106-I8000

80-003057
p. 128 of 129
YHL

Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-003057
Location = TOXL 8164.13.00 06/09
01/18/2011 14:49

Test	AC	Time
01 Diagnostic	OK	14:50
02 Room Air	RFI*	14:50
03 Room Air	0.000	14:50

*Invalid Test
Inhibited - RFI

Sub Name = DISCOVER, THE SPIRIT
Sub DOB = 02/01/1992
Sub Sex = Female Weight = 150
Test = DUI Cit = NA
Dr. Lic. = ND/DIS921456
Simul Ser No = 00000
Std Sol No = 000
County = 08 Oper No. = 777777

I followed the Approved Method and the instructions displayed by the Intoxilyzer in conducting this test.

Deb Kashur
Operator Signature
DEB KASHUR

Remarks:

RFI check

Form 106-18000

80-003057
P. 129 of 129
YFM