



ETHANOL GAS STANDARD CYLINDER REPORT

OFFICE OF ATTORNEY GENERAL

CRIME LABORATORY DIVISION

Toxicology Section/Breath Alcohol Program

SFN 59282 (10-2013)

Chemical Test Operator Name (Print) <i>RAY WEBER</i>		
Location <i>TRAIL CO. S.D (HILL)</i>	Intoxilyzer® Serial Number <i>80-004205</i>	
Gas Lot Number <i>19615080A5</i>	Gas Cylinder Number <i>#32</i>	Gas Expiration Date <i>09/05/2017</i>

Check When Done:

1. Scan/Enter Gas Cylinder Information (Level 1, Function S).

2. Perform an ACA Test (Level 1, Function C).

Write in the ACA Test Results (Reported to 3 Digits, Ex. 0.081).

A. 0. *080* AC

B. 0. *080* AC

C. 0. *080* AC

3. Complete the Top Portion of the Intoxilyzer® Record (SFN50496, Form 120-G) and Place it by the Intoxilyzer® for Use.

4. File Previous Intoxilyzer® Record (SFN50496, Form 120-G) at the Intoxilyzer® Location at the Agency.

5. Send the Following to the Crime Laboratory:

A. Ethanol Gas Standard Cylinder Report (SFN59282, Form 105-G).

B. ACA Test Record.

Chemical Test Operator Signature <i>Ray Weber</i>	Date <i>01/18/2016</i>
Reviewed By (Crime Laboratory Use Only) <i>Deb Shanaver</i>	Date <i>21 Jan 16</i>


Intoxilyzer Test Record and Checklist
NDOAG Crime Lab. Div., Bismarck, ND 58501

CMI, Inc. Intoxilyzer Alcohol Analyzer
North Dakota Model 8000 SN 80-004205
Location = HILL 8164.13.00 06/09
01/18/2016 00:29

DRY CAL CHECK

Test	AC	Time
01 Room Air	0.000	00:30
02 Std. Gas	0.080	00:30
03 Room Air	0.000	00:30
04 Std. Gas	0.080	00:31
05 Room Air	0.000	00:31
06 Std. Gas	0.080	00:32
07 Room Air	0.000	00:32

Lot No = 19615080A5
Cyl No = 32
Exp Date = 09/05/2017
County = 49 Oper No. = 058405



Operator Signature
RAY WEBER

Remarks: